



## Mobile holder for DR detectors, cassettes & GE FlashPad

1330/4/1 (GE #E06731BL) Quickly and accurately position portable DR panels, CR, DR cassettes & GE FlashPad (registered design No.001036594)

This versatile Kenex holder can be used with the GE FlashPad, portable DR panels, CR, DR and film cassettes/grids from 24 x 30 cm to 35 x 43 cm and up to 3.7 cm thick. It rolls smoothly on large quick locking castors and facilitate examinations in accident & emergency departments, operating rooms, and radiographic rooms. The heavy-duty base gives a low centre of gravity, which helps provide a precise, stable, and safe imaging platform. Vertical movements are counterbalanced for simple and quick height adjustments.

The holder is particularly suitable for cross-table views, sometimes with the DR panel or cassette positioned slightly below the tabletop to eliminate image cut-off. It also assists in acquiring fast skyline views of the patellar, or effective views of the shoulder with the patient on the table or in a chair.

The sides of the holder can be quickly extended or retracted to secure different sizes. These telescopic adjustments present a compact platform with no protruding parts. The holder can be turned, tilted, and orientated to suit any desired examination position. It can therefore be placed above, besides, below or beneath a table or trolley. It is adjustable for height from floor level to 129 cm (measured from the lower edge of the DR panel or cassette), and can overhang the x-ray or operating table by up to 62 cm.


### Properties:

- > Stable and compact design holds portable DR panels and cassettes safely in place.
- > Eliminates retakes caused by improvised positioning.
- > Unlimited orientation for universal positioning possibilities.
- > For cassettes/image receptors up to 3.7 cm thick.
- > Telescopic width adjustment for cassettes/image receptors from 24.5 cm to 53.5 cm.
- > Height adjustable from floor level to 129 cm.
- > Overhangs table or trolley by up to 62 cm.
- > Counterbalanced for easy raising or lowering.
- > For weights < 4.2 kg
- > Twin wheel 12.5 cm diameter castors, with combined swivel/rolling brakes, have cable push-away guards.
- > Smooth easy to clean hardwearing surfaces.
- > Base measurements 67 cm deep x 61 cm wide.
- > Height measurements 179 cm.

**Applications include Radiology, Urology, Surgery and Emergency Rooms.**



Telescopic width adjustment

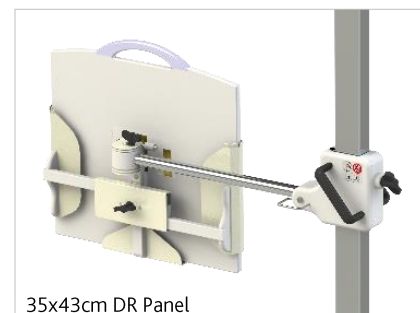

35x43cm DR Panel



24x30cm Cassette



Kenex holders are designed & manufactured in compliance with the requirements for safety and essential performance of medical equipment. The instability test requires that all mobile equipment should remain stable when tilted at an angle of no more than 5°. Even when supporting a load at the maximum height, this Kenex holder will not overbalance until tilted at 22°. This feature ensures that expensive DR panels or other image receptors can be used and positioned without the risk of the unit tipping over.

### Version


- > **1330/4/1** Counterbalance for a 4.2 kg weight and generally used for lighter & thinner cassette size portable DR detectors and CR or film cassettes. Net weight: 55 kg

### Weights & dimensions

Transport packaging details: -


Box 1: Export packed base, 78 x 74 x 36 cm, 45 kg

Box 2: Export packed column & holder, 204 x 41 x 37 cm, 22 kg

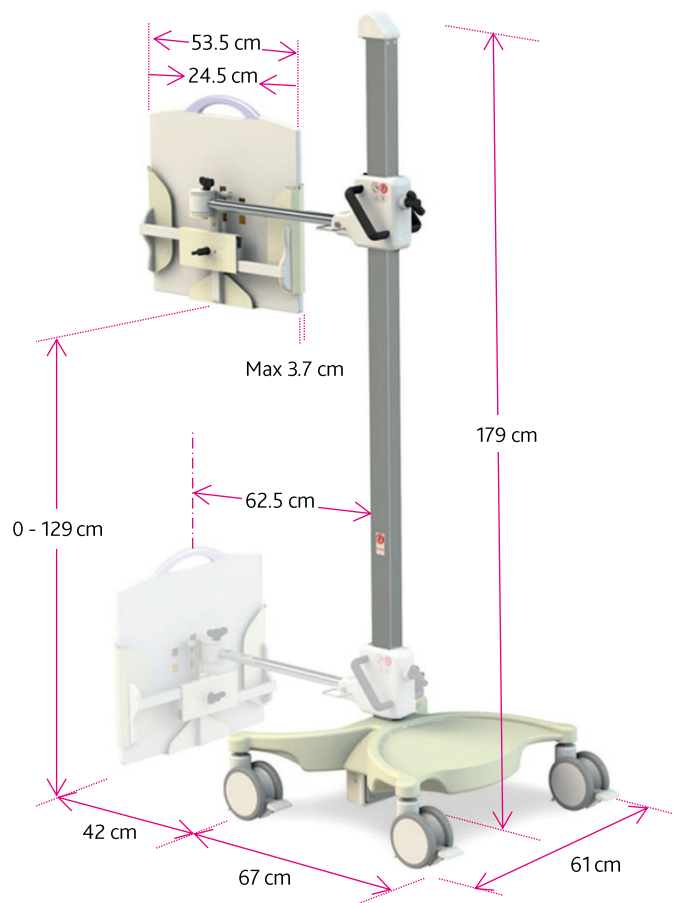

### Positioning possibilities

Stable view from any angle with unlimited orientation for universal positioning

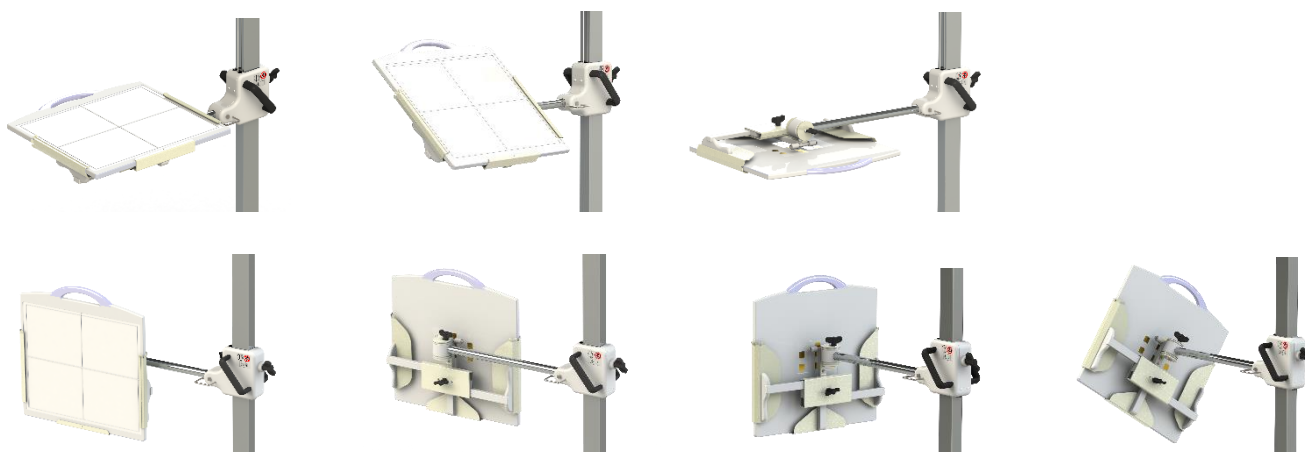

#### Quality management systems and certification

Kenex is certified to ISO 13485 & ISO 9001 for the design & manufacture of medical devices and meets the requirements of EU Regulation 2017/745 (MDR) regarding medical devices and EU Regulation 2016/425 (PPE) regarding personal protective equipment. Kenex has in place methodology for identifying, addressing, and evaluating risk in accordance with standard ISO 14971. All Kenex products are CE certified. Copyright © Kenex (Electro-Medical) Limited. All rights reserved.

