IA001

1305/8/1, 1305/3/1

1305/8/1

2 - 18, 22 - 23

1305/3/1

2 - 14, 19 - 23

EN- Instructions for use in English

Safety symbols

WARNING

Denotes a hazard with a medium level of risk, which, if not avoided, may result in serious injury or injury requiring treatment.

CAUTION

Denotes a hazard with a low level of risk, which, if not avoided, may result in minor or moderate injury.

Symbols on products

CAUTION

Do not lean against or push the equipment with the brakes activated as it may overbalance.

CAUTION

With counter balanced models always hold the horizontal section or handles to safely control movement. If the load is removed with handles unlocked it will move up rapidly, especially from a low level.

Information for safe use

Intended use

Mobile holders are designed to securely accommodate portable image receptors. The holder provides adjustable height and flexible positioning for optimal placement.

Important information for use

- Ensure the lower edge of the holder is positioned just above the floor for transportation purposes. Care should be taken when moving the equipment.

- Some models have castors fitted with cable guards to prevent them from running over and damaging cables. These guards can also strike a small step or floor protrusion so care should be taken when moving this equipment.
- For use by professional personnel.
- The user must check the device is functioning correctly before use.
- Retain these instructions for future reference.
- Life expectancy 10 years with correct use and care.

Routine checks

- If a device is in collision, or is not free moving, stop using it and have it inspected.

Cleaning

All devices should be cleaned using a mild soap solution or a regular hand dishwashing product. Do not spray on – wipe on with a damp cloth.

Reporting obligation

Any serious incident that occurs concerning these devices should be reported to Kenex and the local competent authority for medical devices.

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











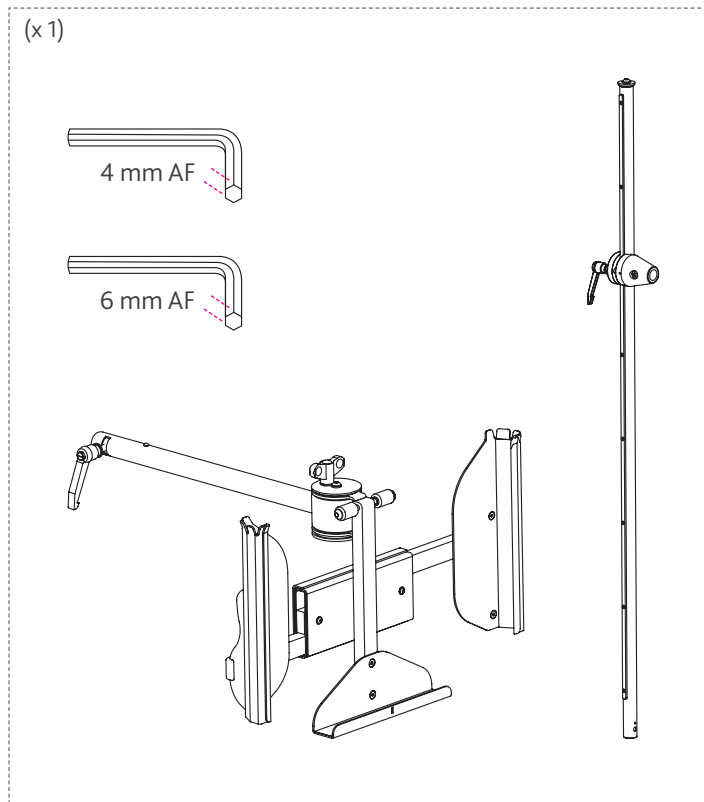
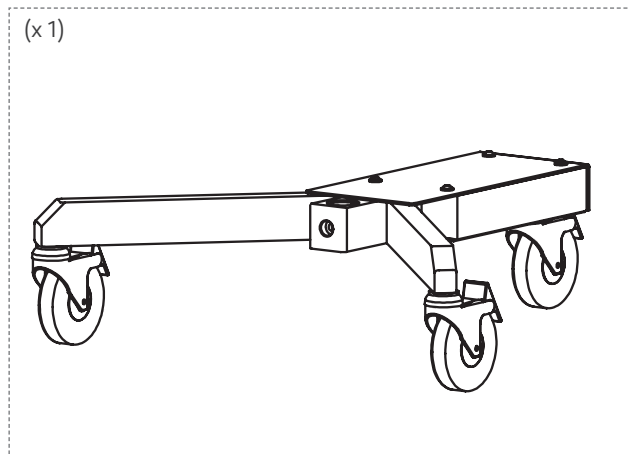
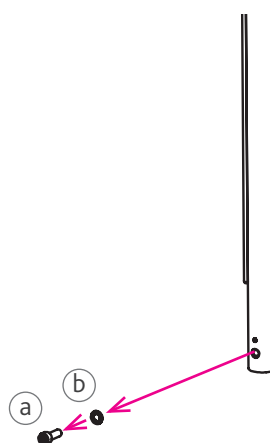
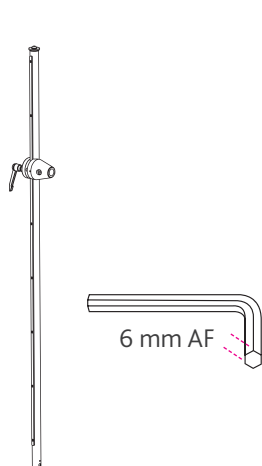
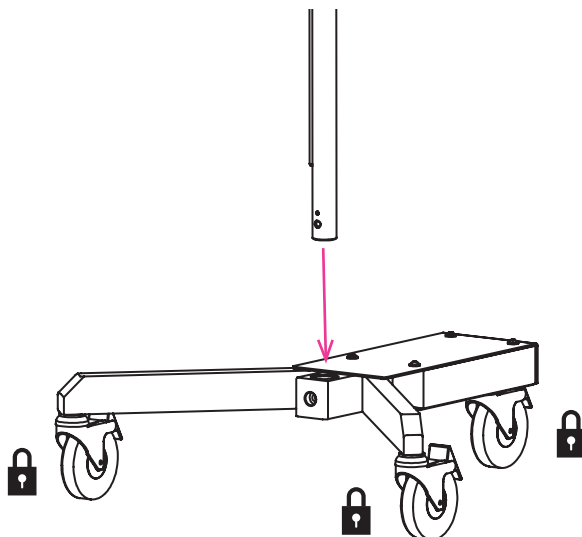
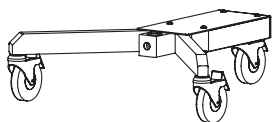
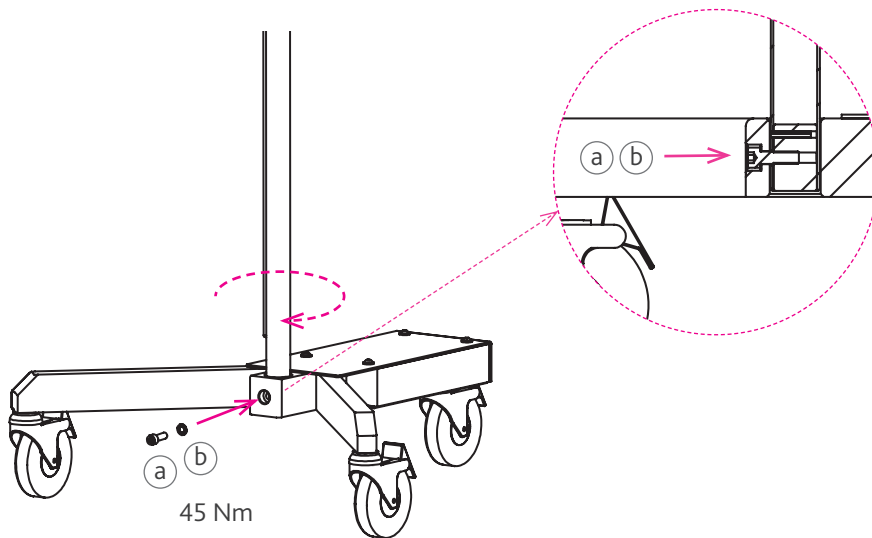
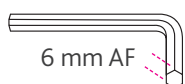
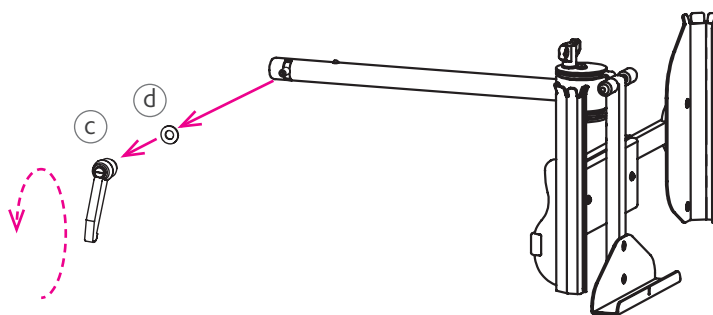
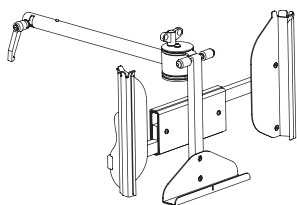
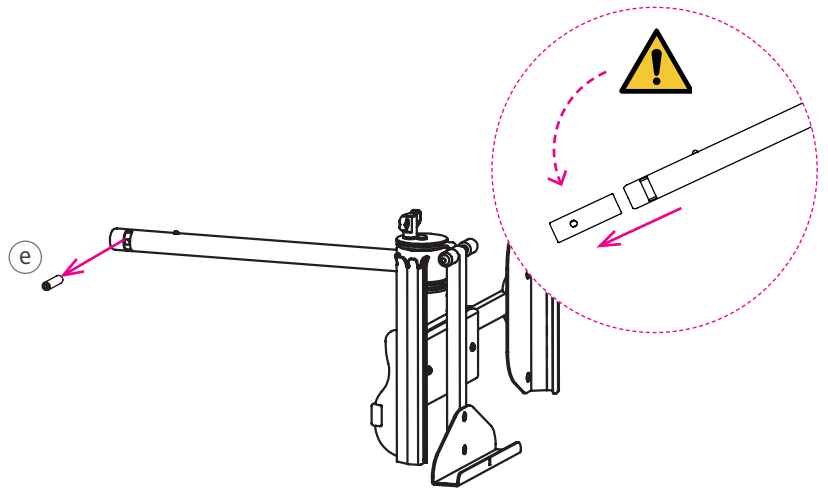
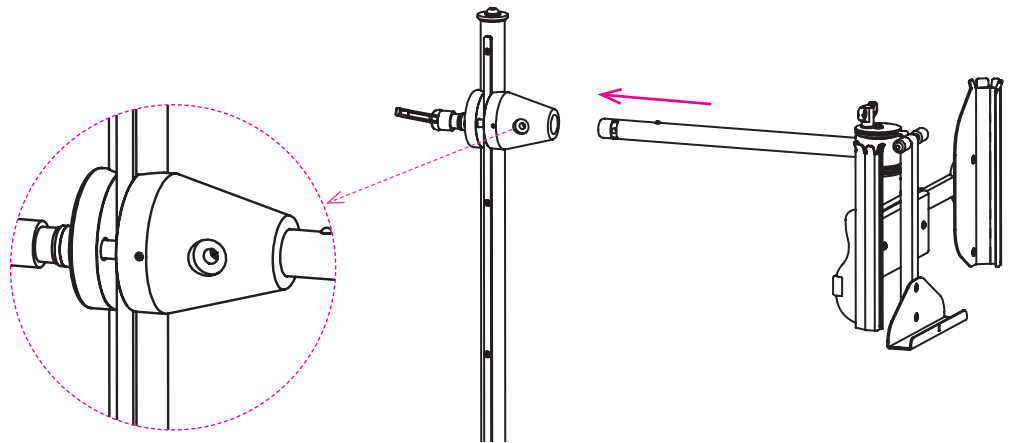
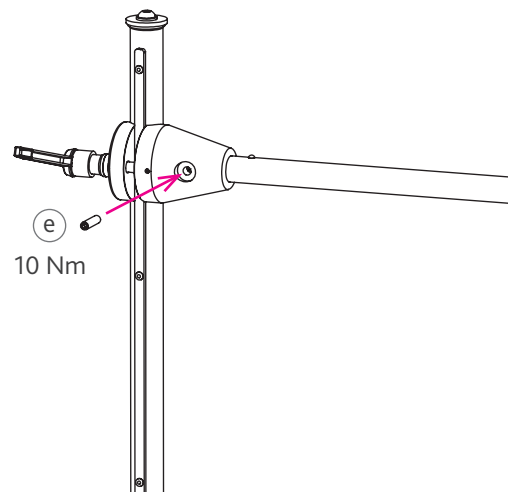
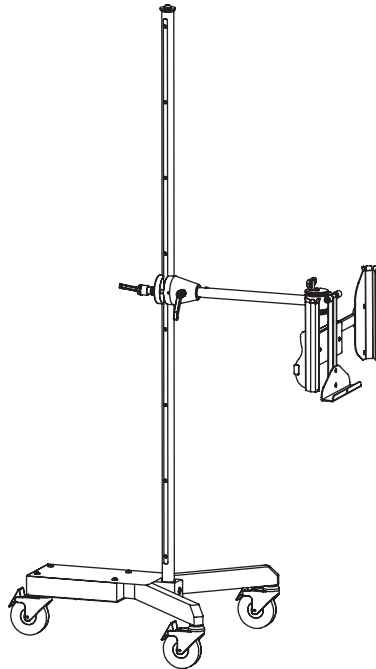
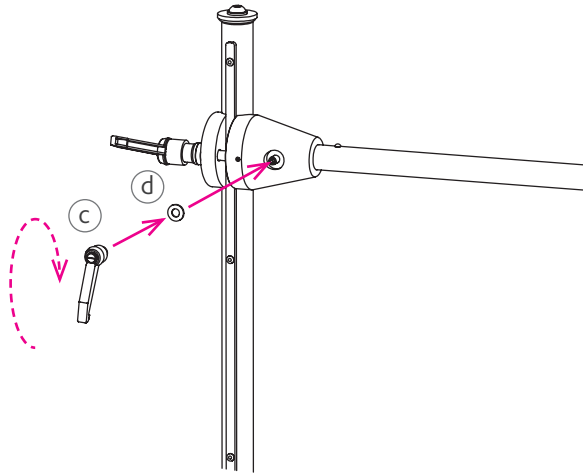
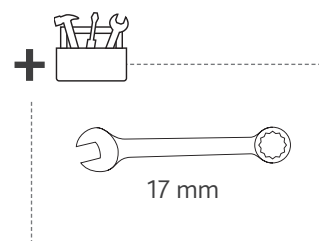
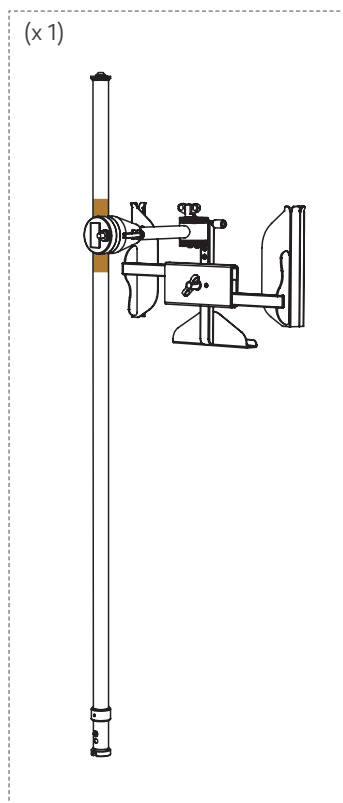
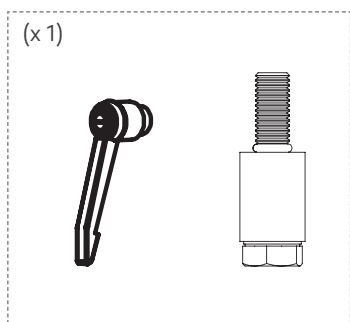
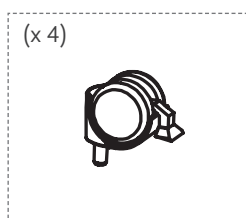
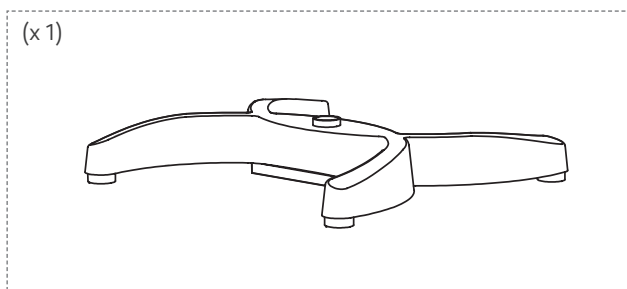
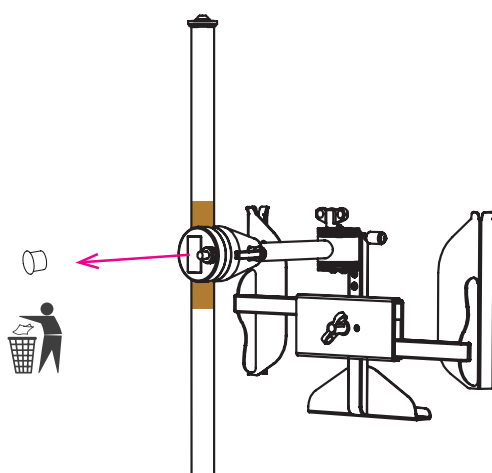
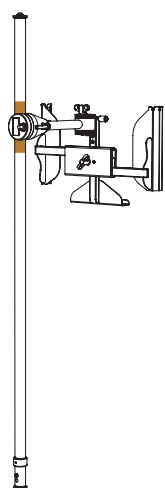
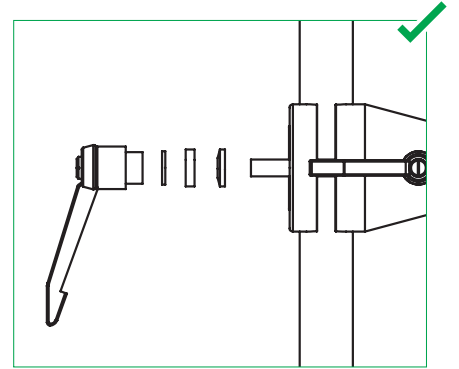
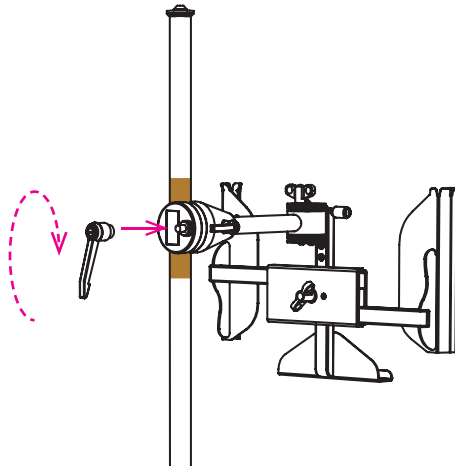
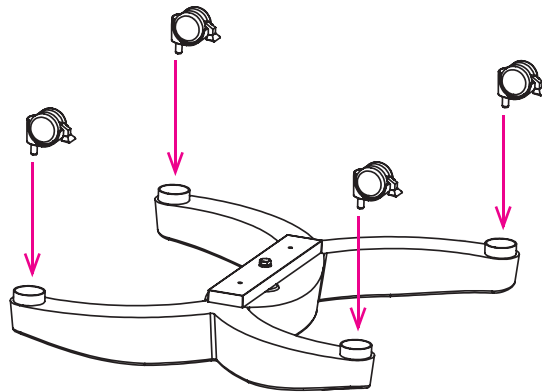
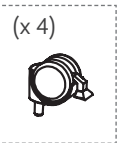
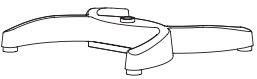
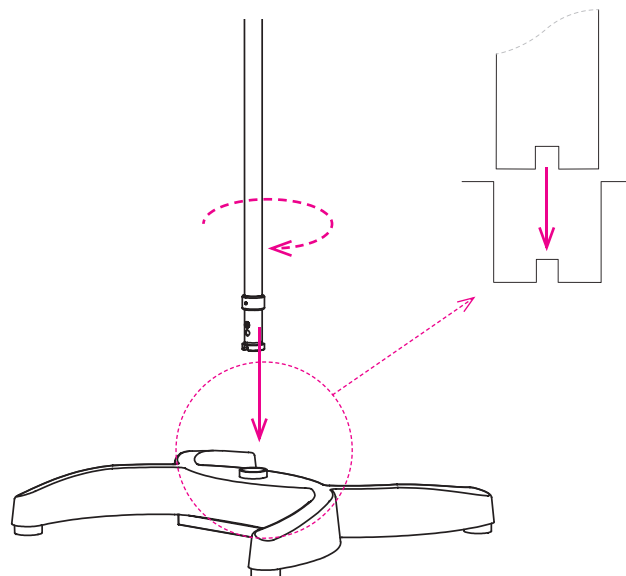
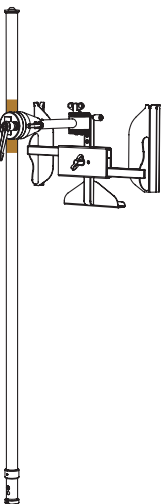
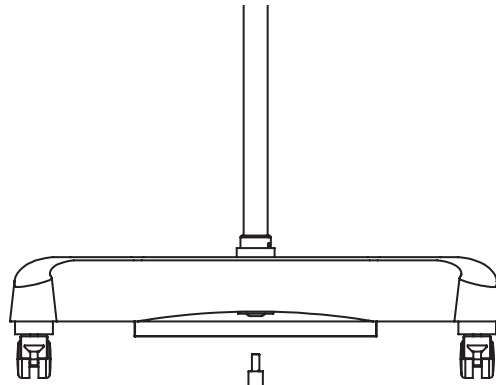
45 Nm

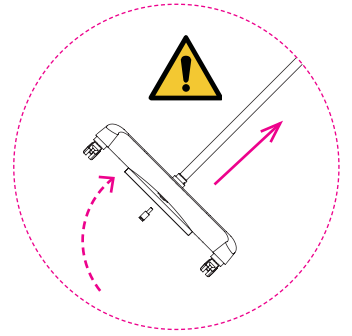6/6

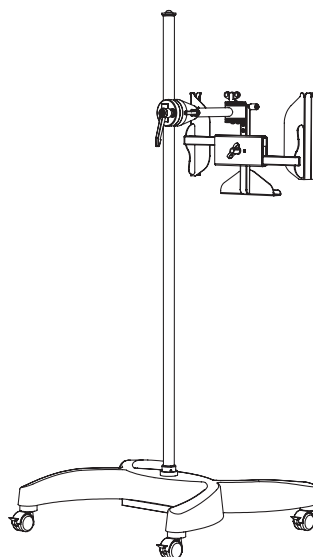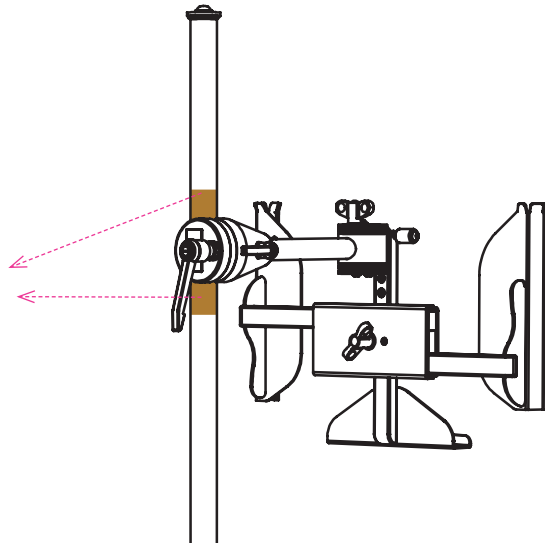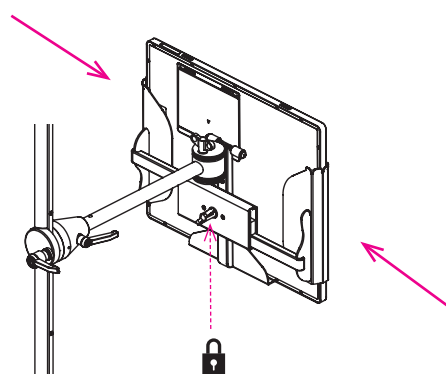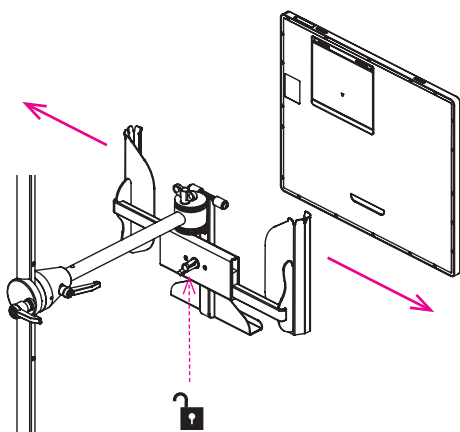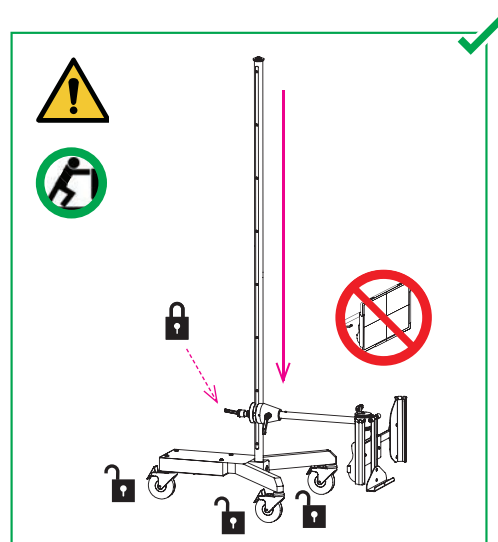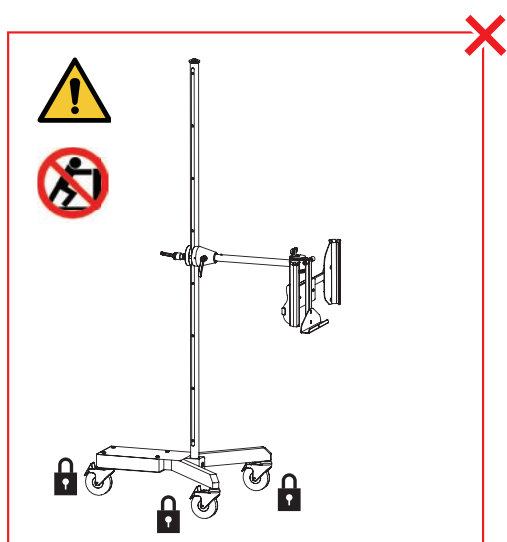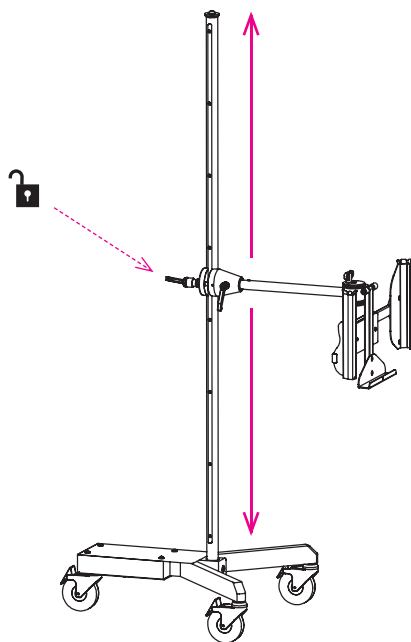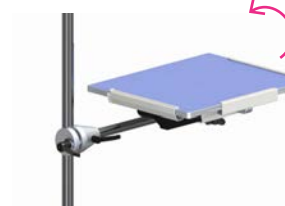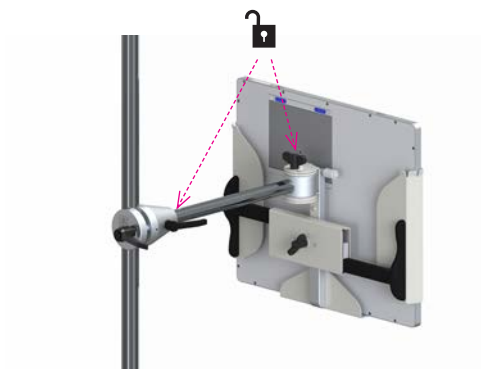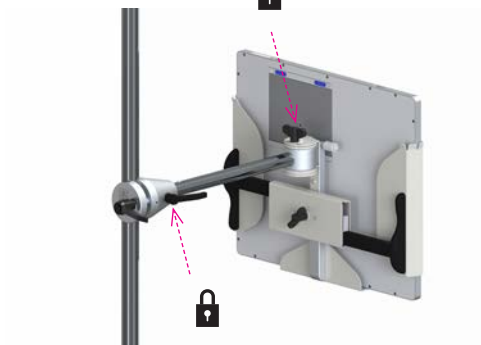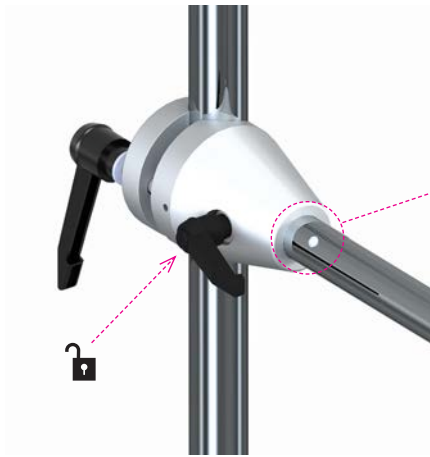Quality management systems and certification

Kenex is compliant and certified according to the following regulations and standards regarding design and manufacture of medical devices: -

2017/745 Medical device regulation (MDR)
2016/425 Personal protective equipment (PPE)
BS EN ISO 13485:2016 (Quality management systems for medical devices)
BS EN 61331-1:2014 (Determination of x-ray attenuation properties)
BS EN 61331-2:2014 (Translucent protective plates)
BS EN 61331-3:2014 (X-ray protective clothing)
BS EN 60601-1:2021 (Medical electrical equipment – basic safety and essential performance)

Quality management systems audited by SGS UK Limited.

Kenex (Electro-Medical) Limited
Unit 17, RO24 Greenway
Harlow Business Park
Harlow, Essex
CM19 5QB
England

T: +44 (0)1279 417241
E: kenex@kenex.co.uk
www.kenex.co.uk

EC REP

EU authorised representative:

Emergo Europe, Westervoortsedijk 60, 6827 AT Arnhem,
The Netherlands.

Emergo Single Registration Number (SRN): NL-AR-000000116

Kenex Single Registration Number (SRN): GB-MF-000009022

CH REP

Swiss authorised representative:

MedEnvoy Switzerland, Gotthardstrasse 28, 6302 Zug, Switzerland.

Australian Sponsor

Emergo Australia, Level 20, Tower II, Darling Park,
201 Sussex Street, Sydney, NSW 2000, Australia.

US FDA AGENT

Emergo Global Representation LLC, 2500 Bee Cave Road, Building I,
Suite 300, Austin, Texas 78746, USA.