**310/B-3, 310/B-3/31, 310/B-3/91** (317/05-01, 310/3.1, 310/9.1)  
**326/05**  
**314/05-01, 314/05-02**  
**303**

---

IM001

310/B-3 & 317/05-01

326/05

314/05-02

303/B2/90

## **310/B-3, 310/B-3/31, 310/B-3/91**

---

**310/B-3, 310/B-3/31, 310/B-3/91**  
3 - 21

**317/05-01**  
3 - 17, 22 - 24

**310/3.1**  
3 - 17, 25

**310/9.1**  
3 - 17, 26

---

## **326/05**

---

3 - 17, 27 - 31

---

## **314/05-01, 314/05-02**

---

**314/05-01**  
3 - 17, 32 - 35

**314/05-02**  
3 - 17, 32 - 34, 36 - 37

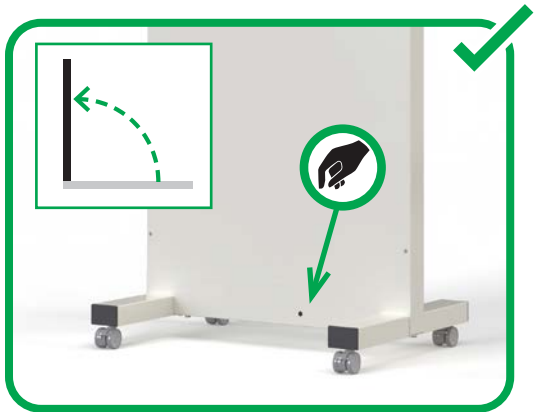
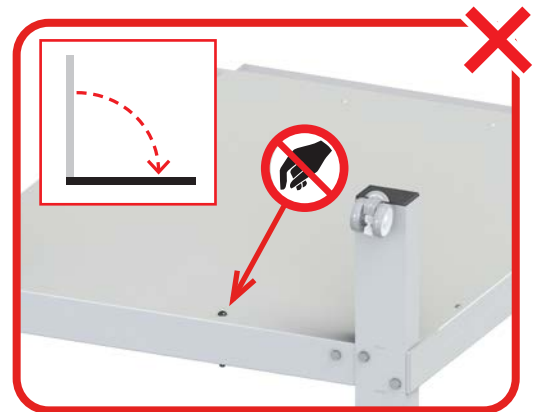
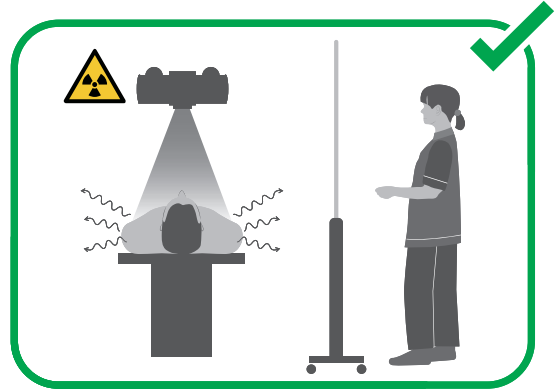
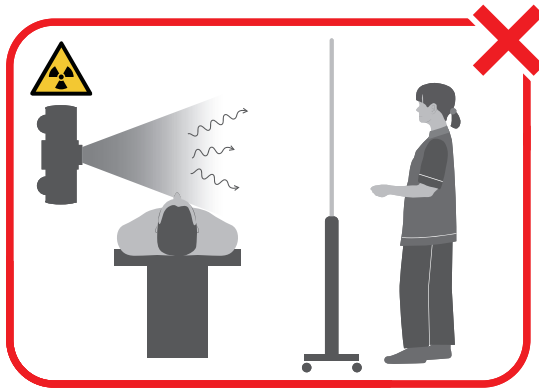
---

## **303**

---

3 - 17, 38 - 39

## EN- Instructions for use in English

### Safety symbols

#### WARNING

Denotes a hazard with a medium level of risk, which, if not avoided, may result in serious injury or injury requiring treatment.

#### CAUTION

Denotes a hazard with a low level of risk, which, if not avoided, may result in minor or moderate injury.

#### WARNING - IONIZING RADIATION

To warn of ionizing radiation.

#### DO NOT REMOVE

Follow instructions carefully and only remove when instructed.

#### FAST MOVING PARTS

Movement with fast speed in direction of arrow from a fixed position

### Symbols on products

#### CAUTION

The pole/curtain will move up rapidly (especially from a low level) if the handle is released. Always hold the top of the vertical pole, or horizontal curtain support to safely control vertical movement.

### Information for safe use

#### Intended use

Protective mobile shielding products are used during diagnostic X-ray procedures to protect healthcare personnel from exposure to scatter radiation.

#### Important information for use

- For use by professional personnel.
- The user must check the device is functioning correctly before use.
- Retain and use the transit screw when moving shields longer distances.
- Retain these instructions for future reference.
- Life expectancy 10 years with correct use and care.

#### Assembly

- Care should be taken when lifting some mobile shields due to the weights involved. Where indicated two people are advised.
- Ensure castors and screws are fitted correctly with washers in the correct order. Refer to the following assembly instructions.

- Do not remove the transit screw until the mobile shield is in the upright position. Failure to do so could result in parts moving with speed and force.

#### Routine checks

- If a device is in collision, or is not free moving, stop using it and have it inspected.

#### Cleaning

All devices should be cleaned using a mild soap solution or a regular hand dishwashing product. Do not spray on – wipe on with a damp cloth. Ensure liquid materials do not dry on lead glass / lead acrylic transparent shields as this can cause damage.

#### Reporting obligation

Any serious incident that occurs concerning these devices should be reported to Kenex and the local competent authority for medical devices.

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









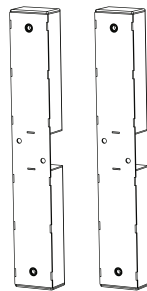
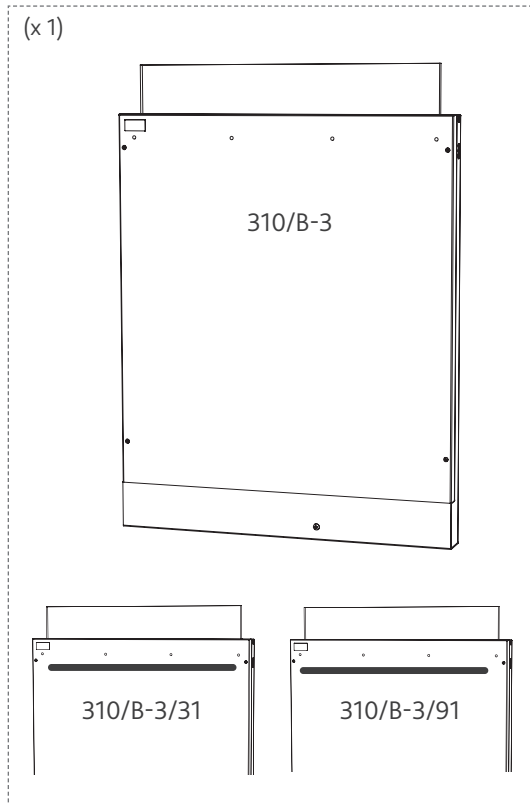
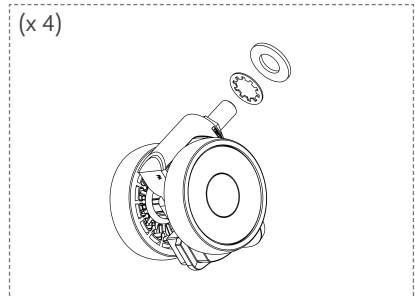
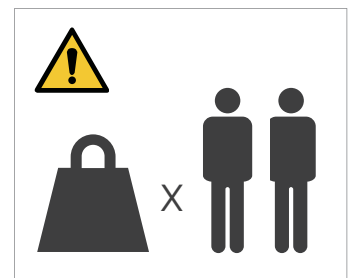
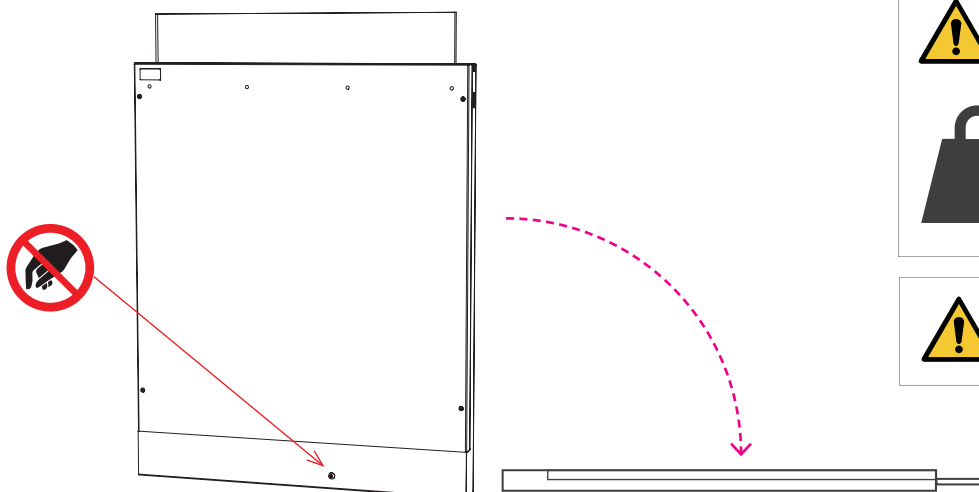
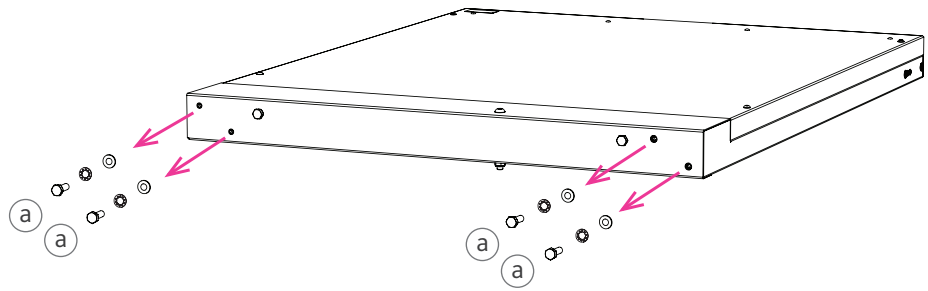
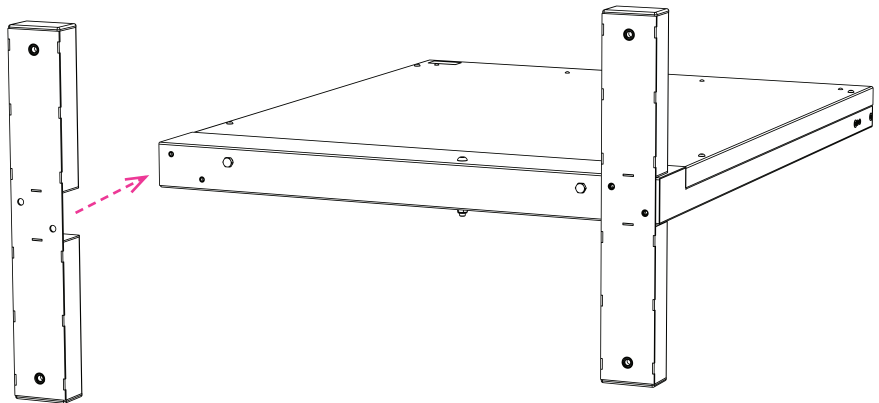
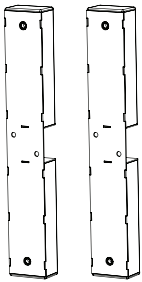
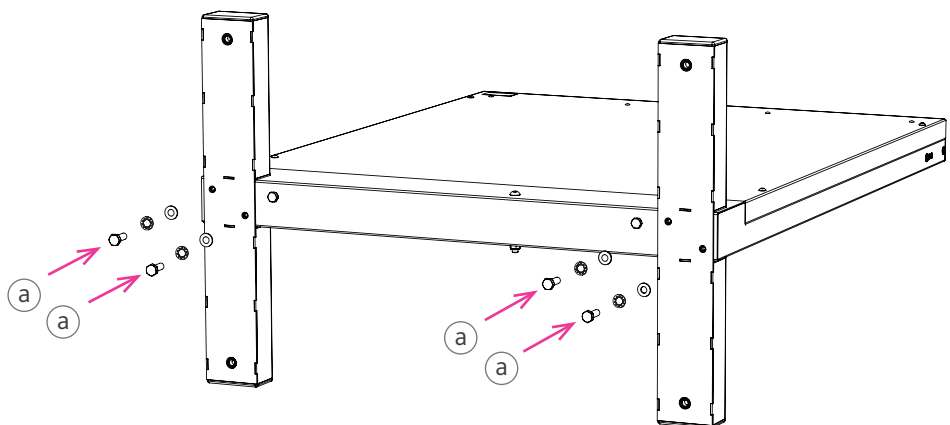
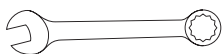
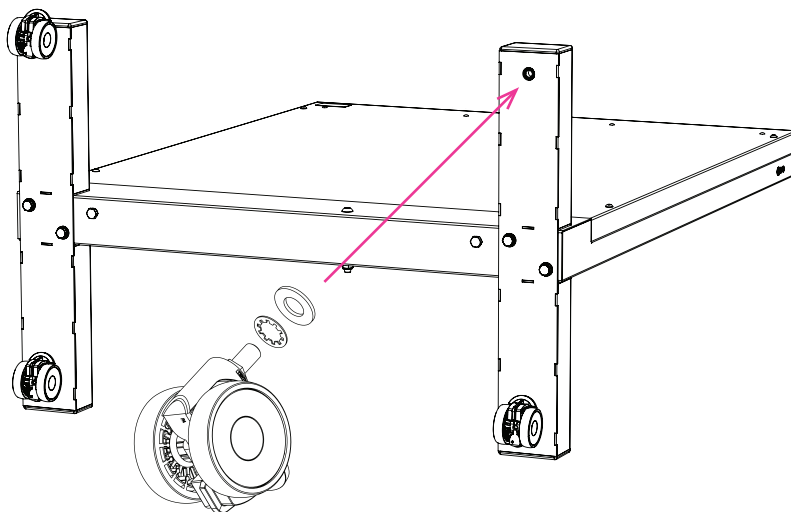
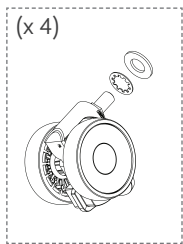
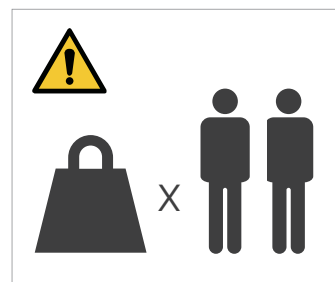
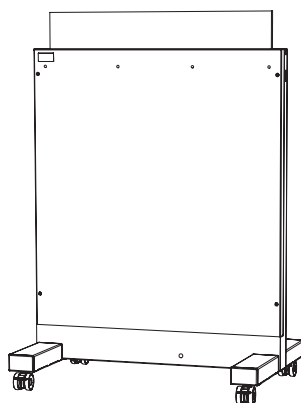
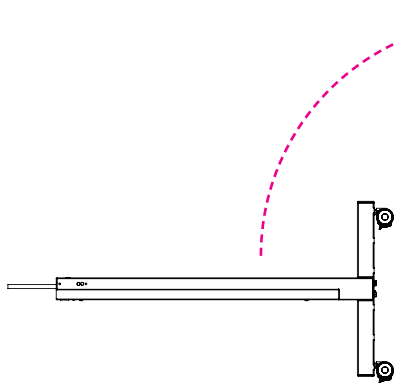
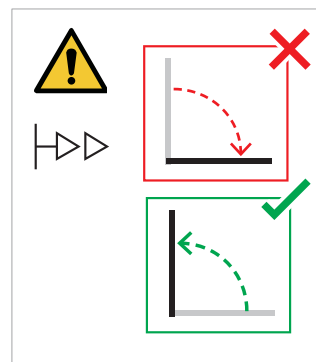
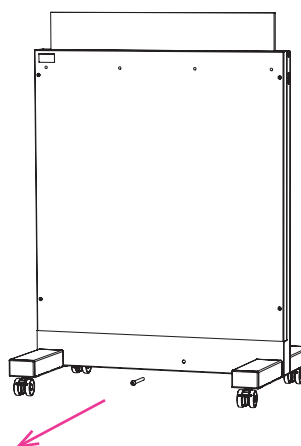
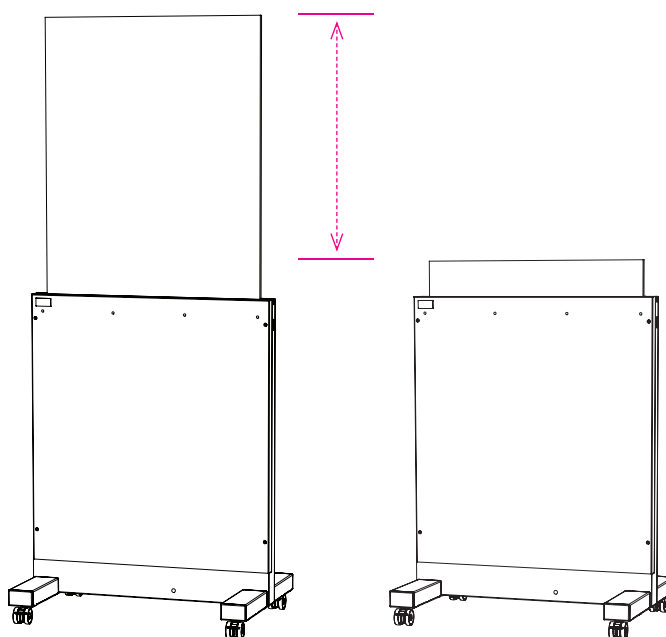
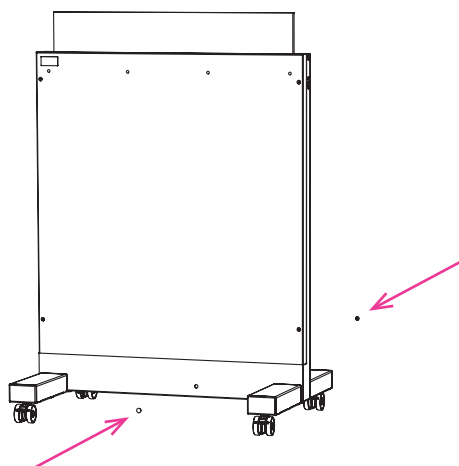
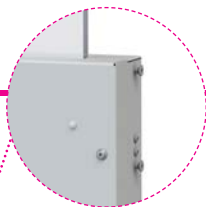
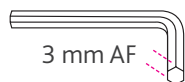
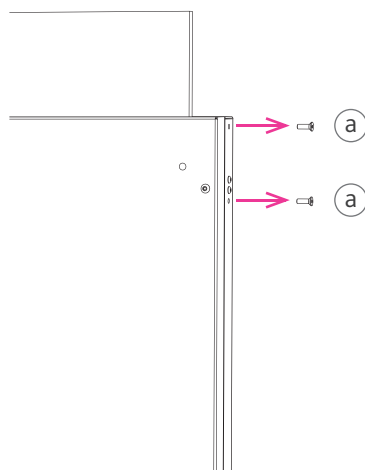
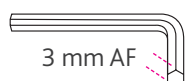
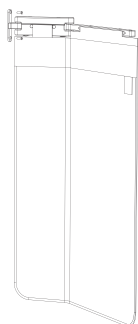
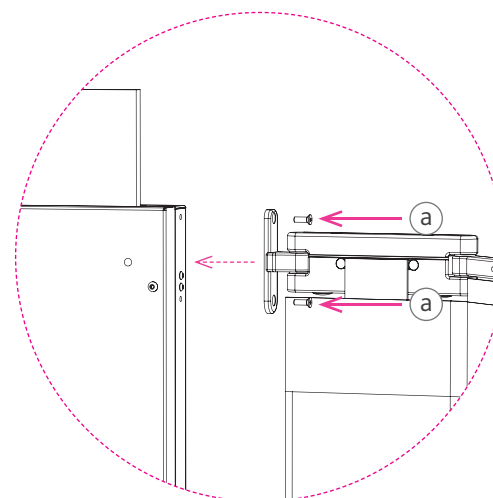
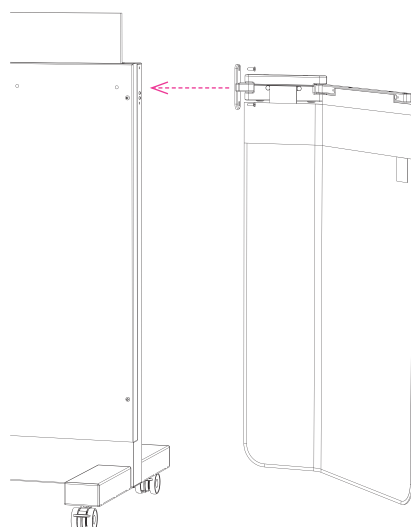
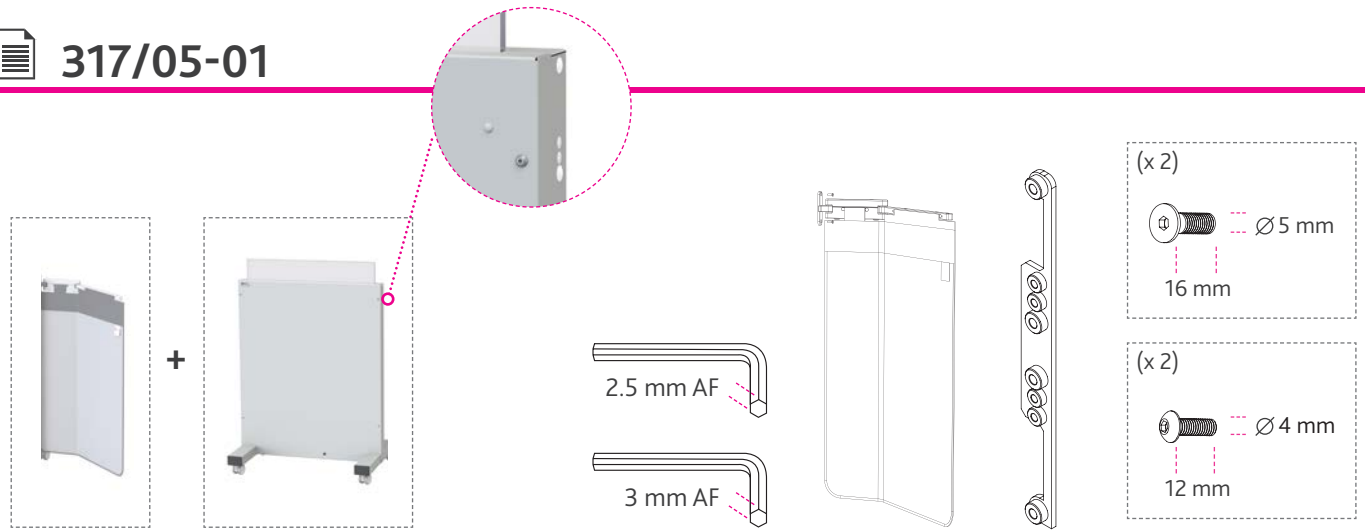
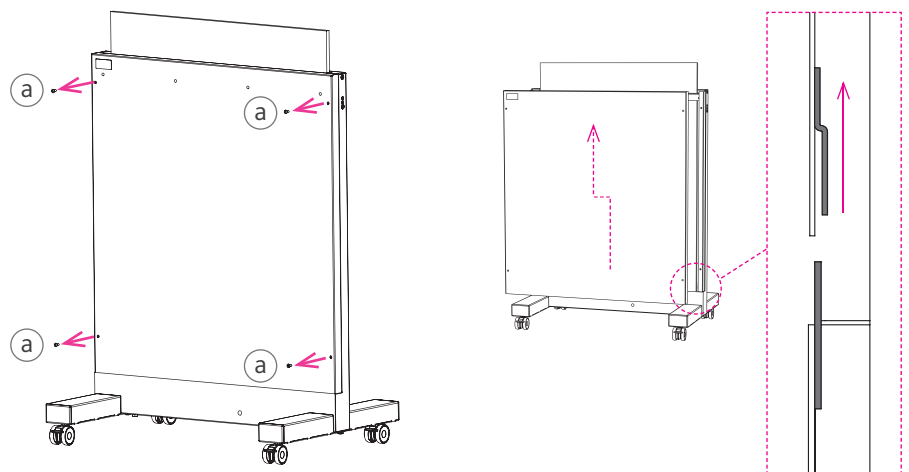
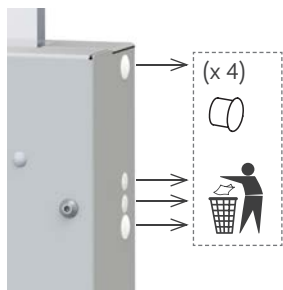
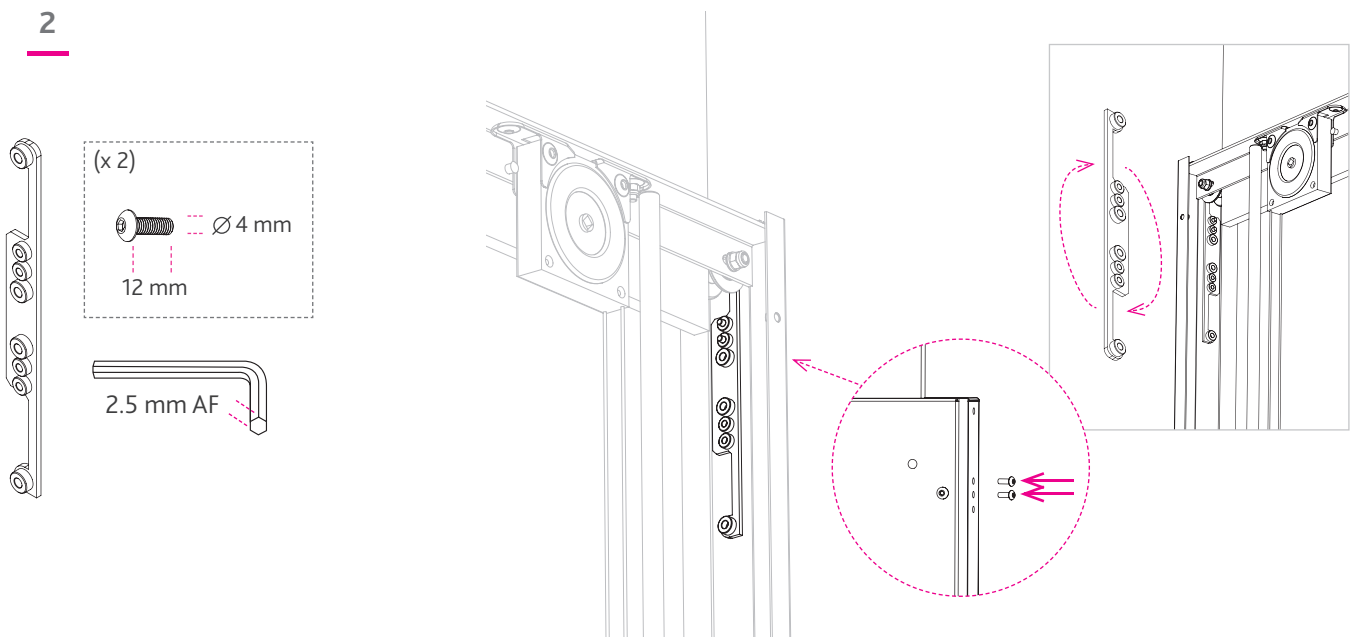
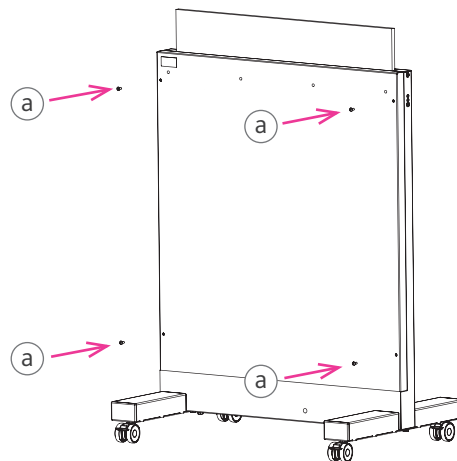
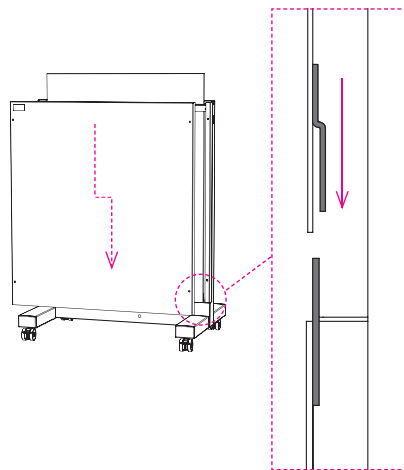
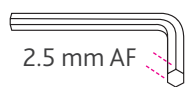
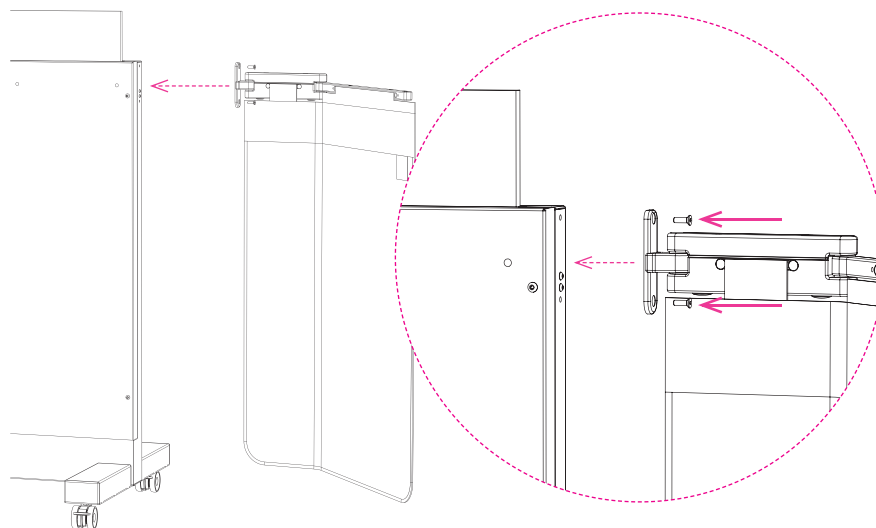
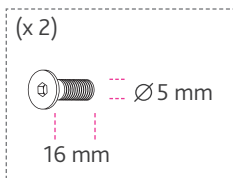
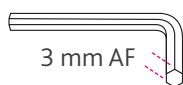
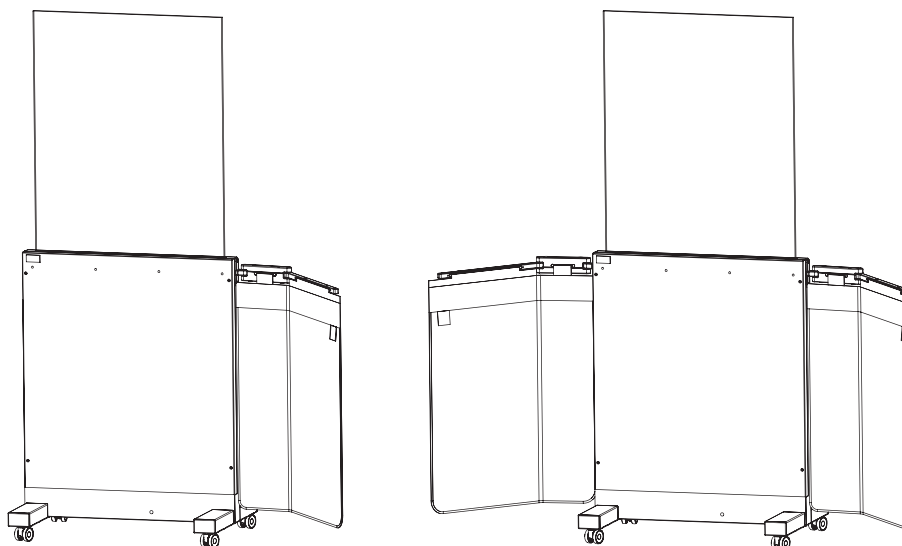
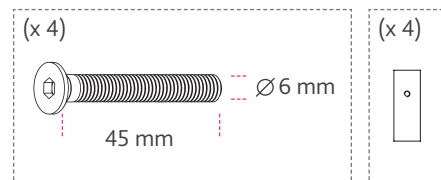
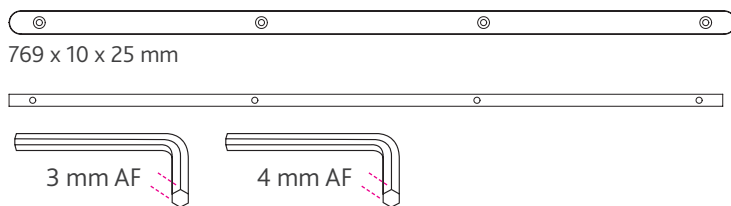
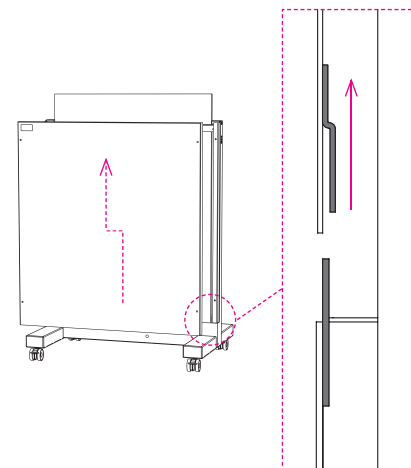
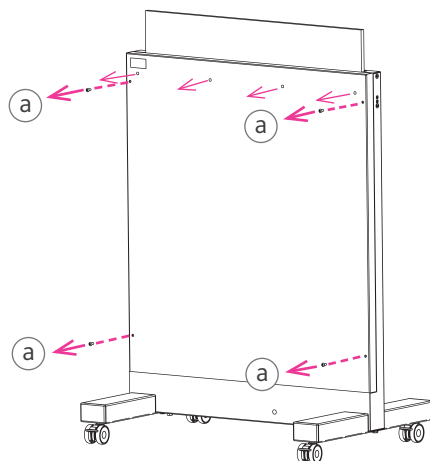
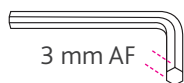
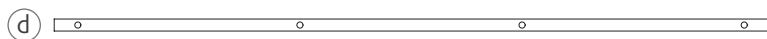
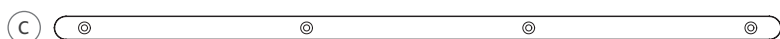
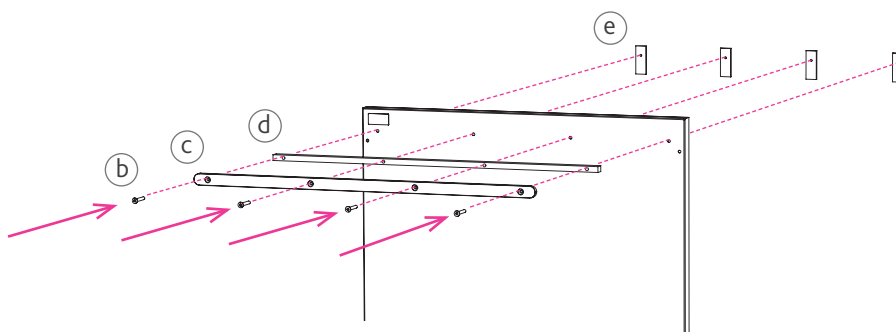
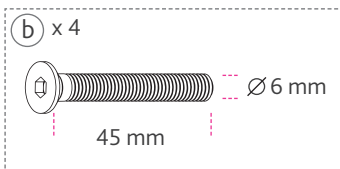
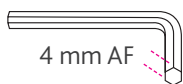
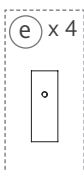
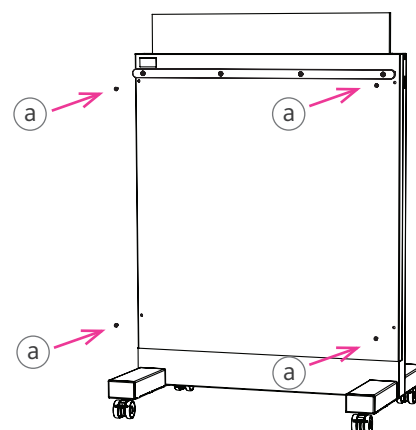
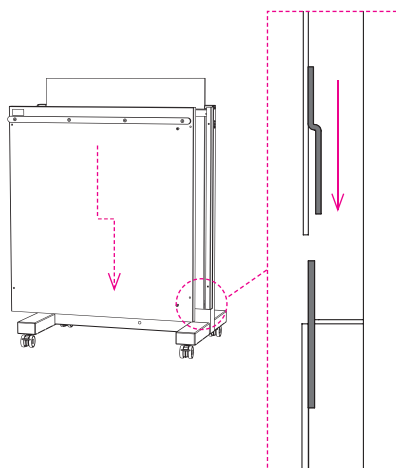
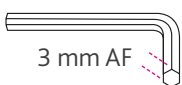
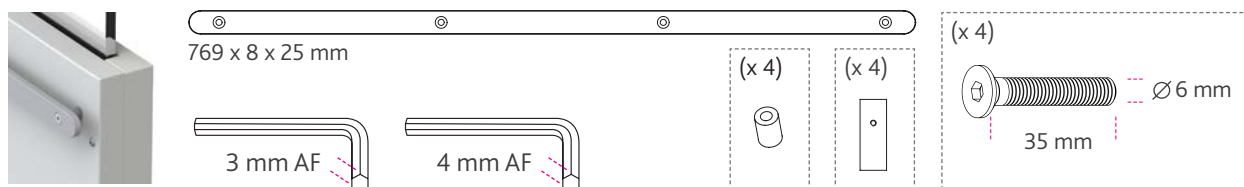
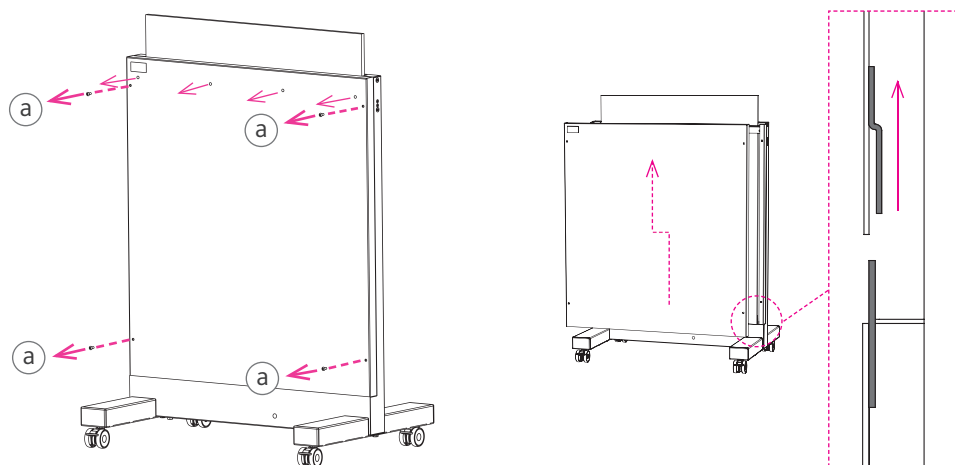
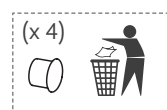
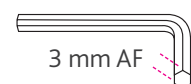
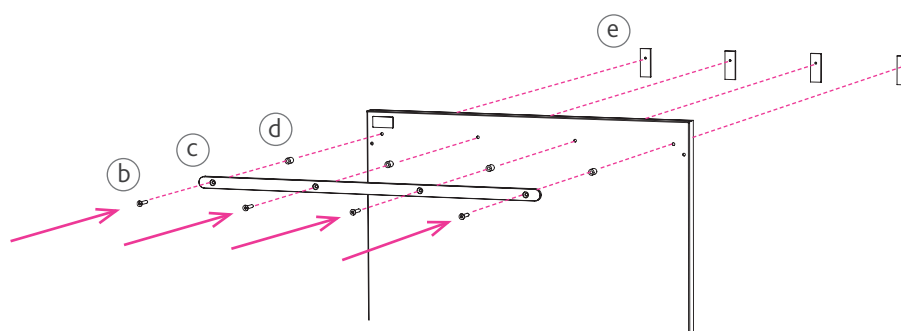
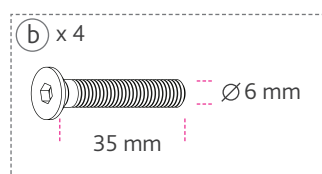
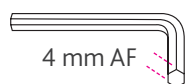
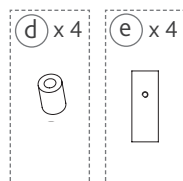
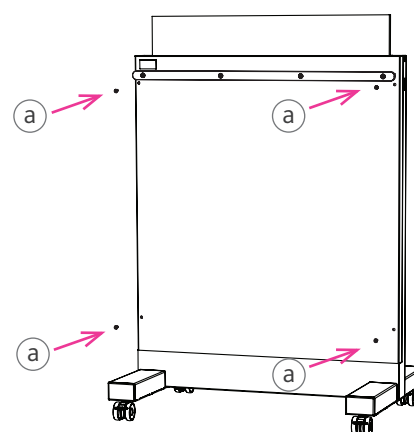
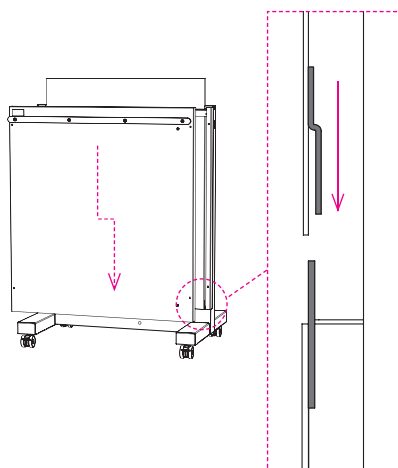
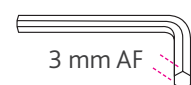
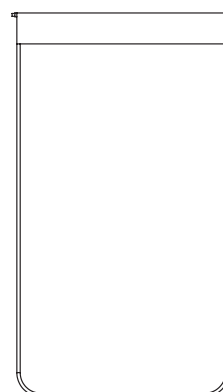
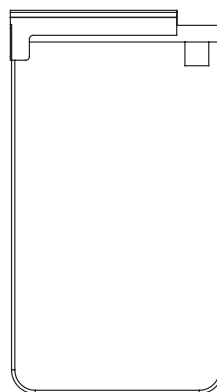
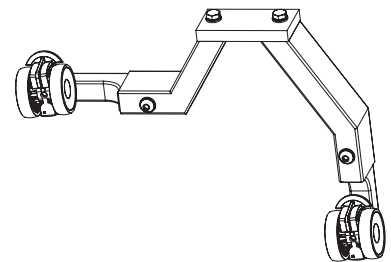
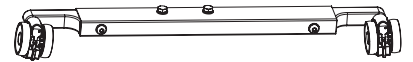
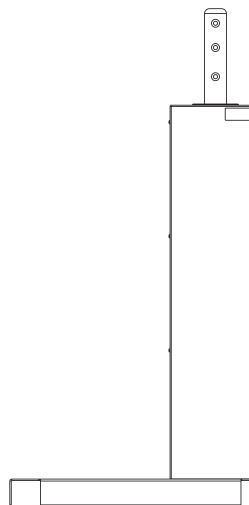
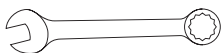
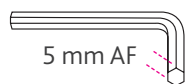
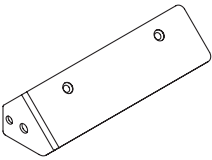
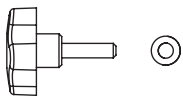
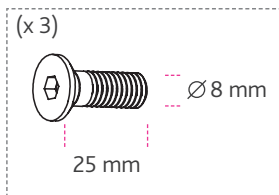
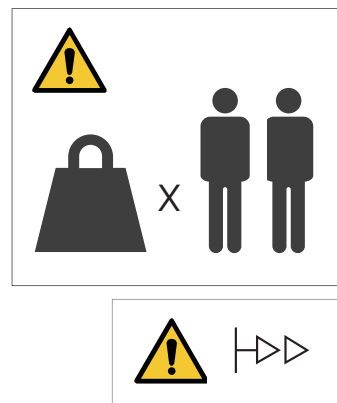
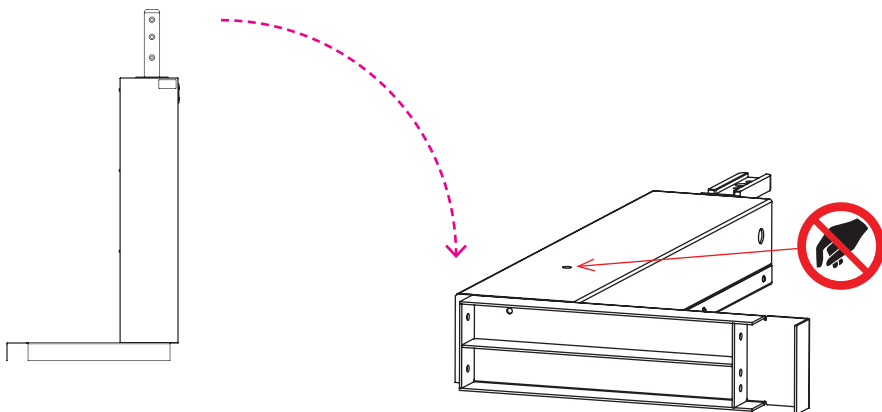
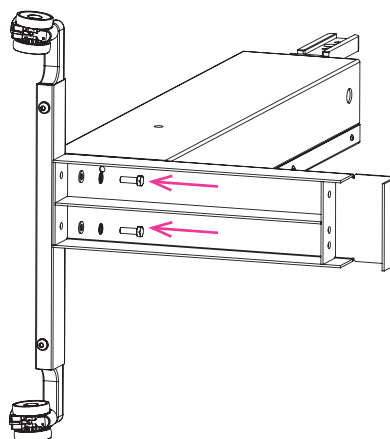
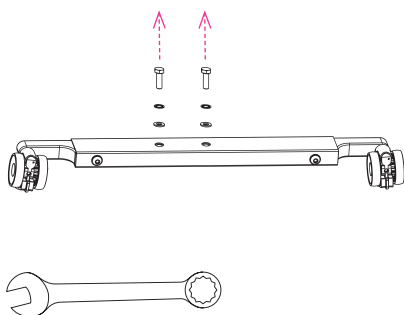
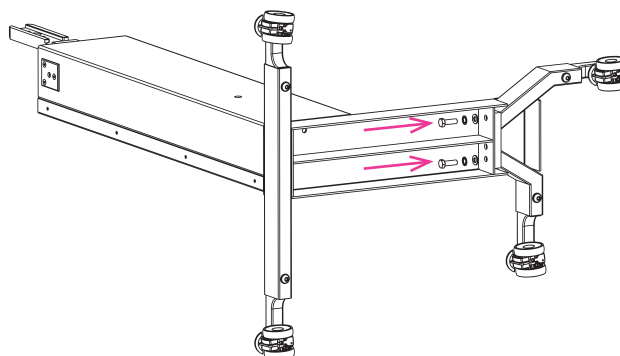
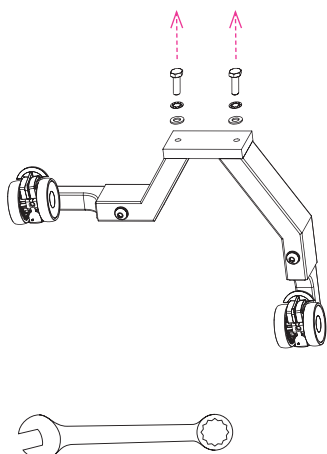
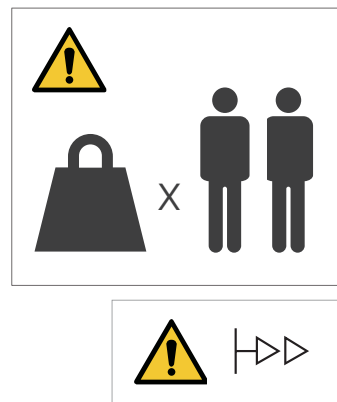
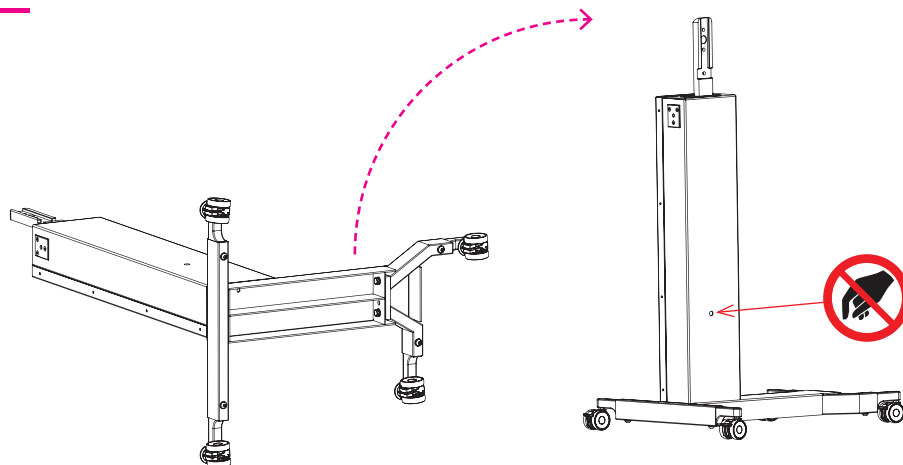
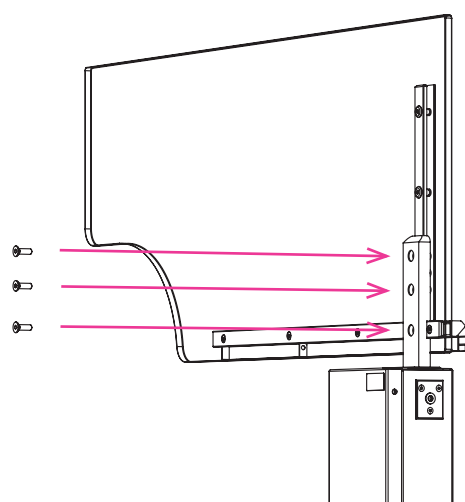
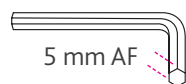
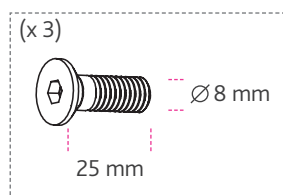
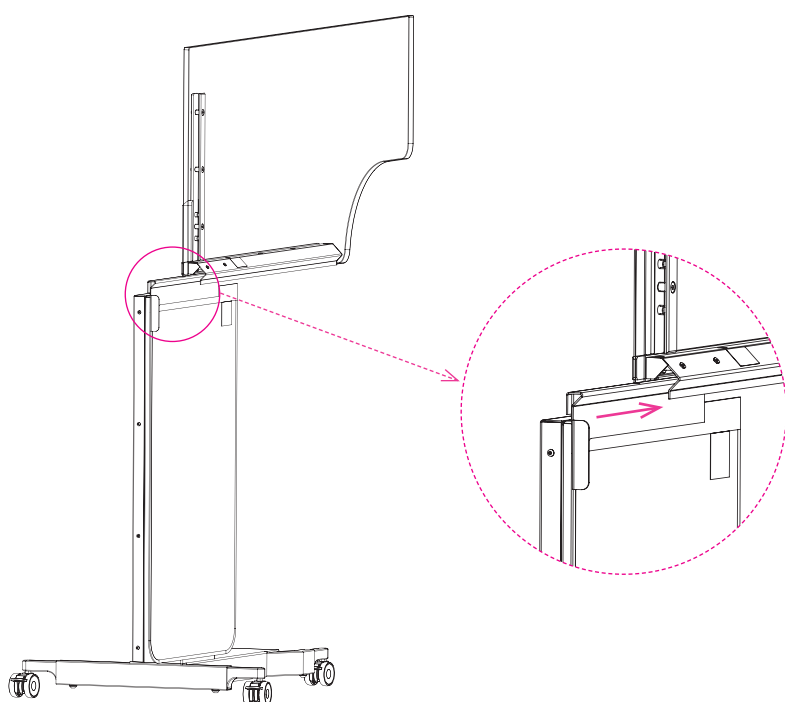
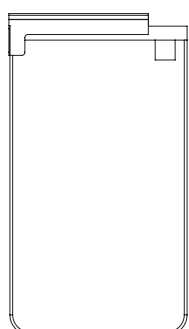
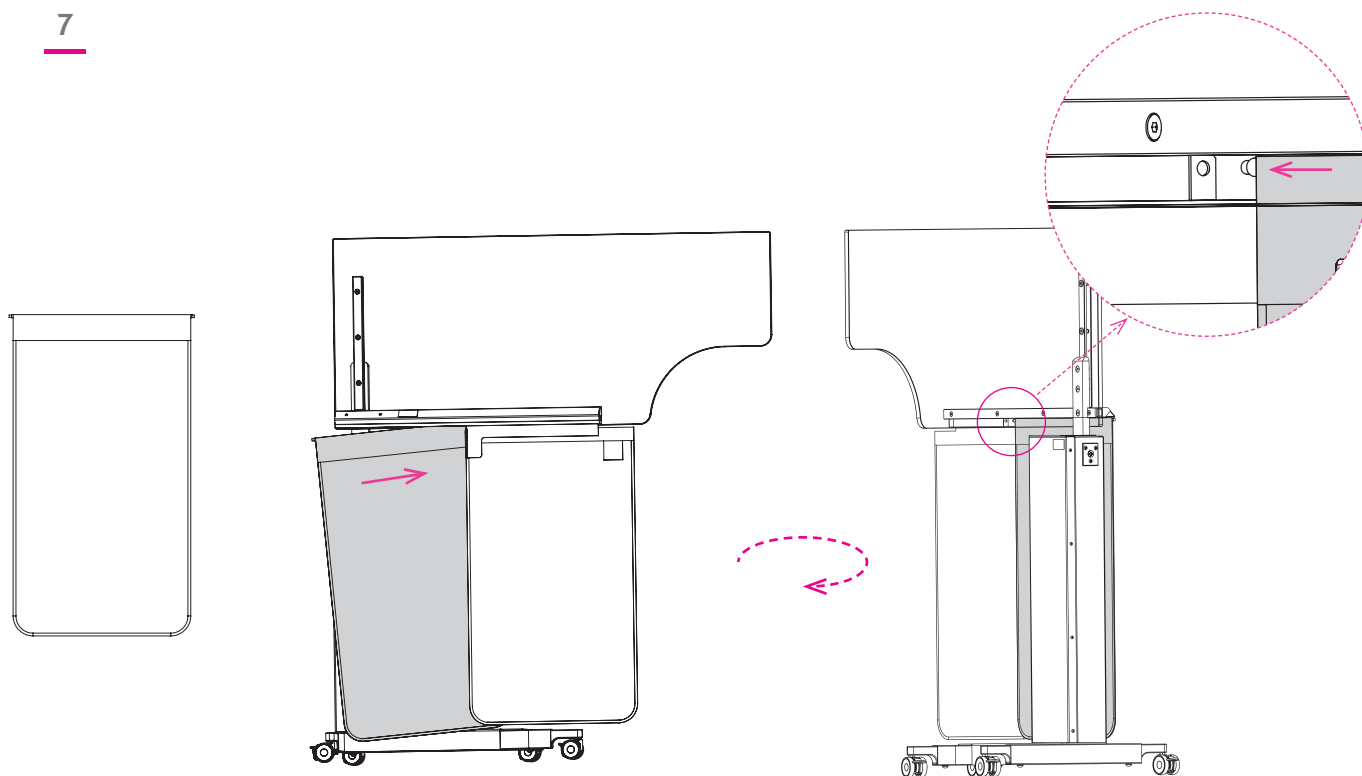
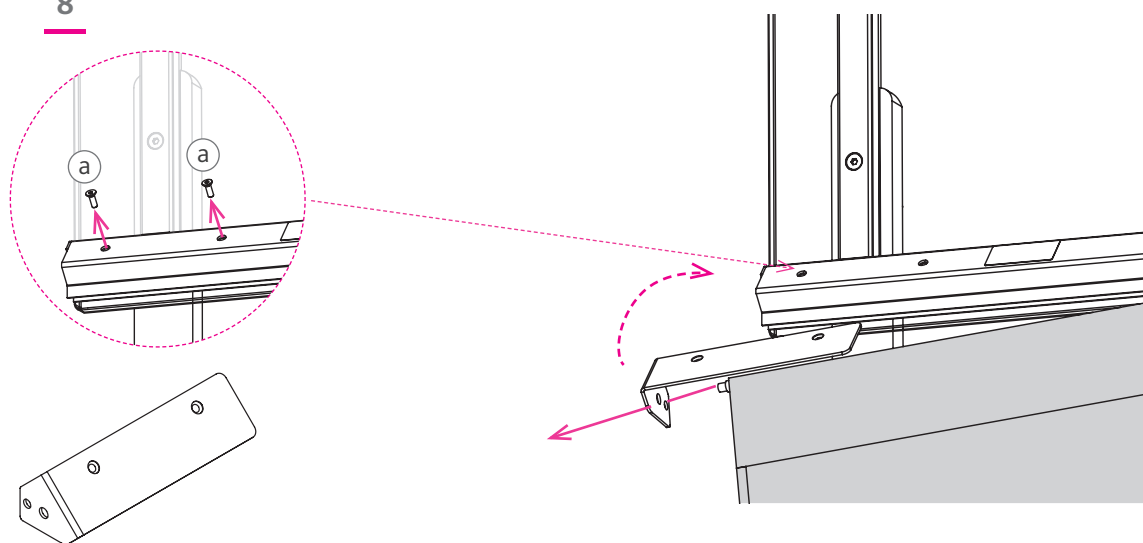
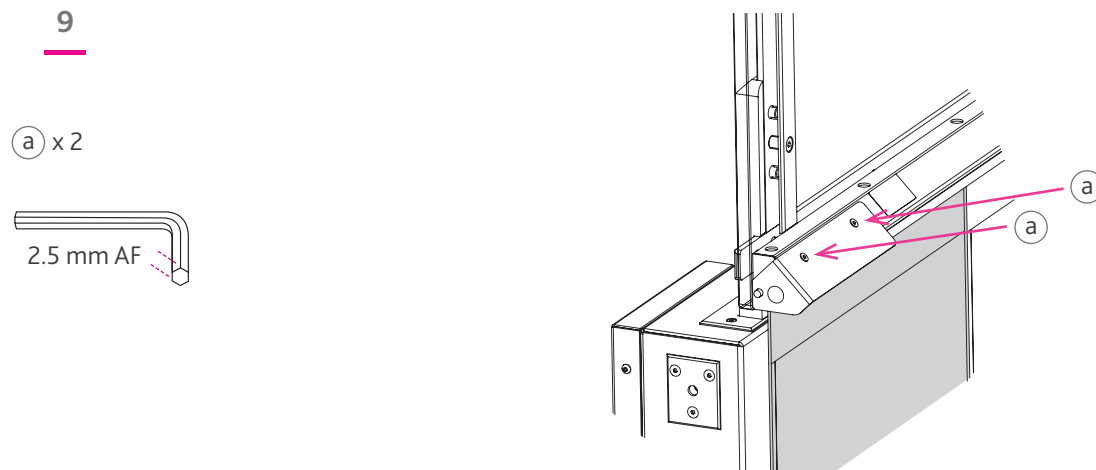
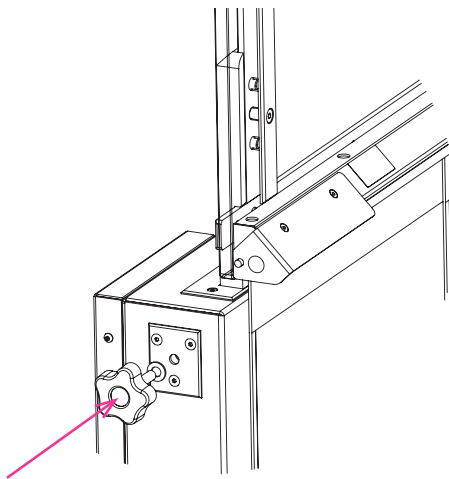
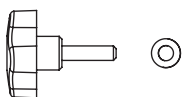
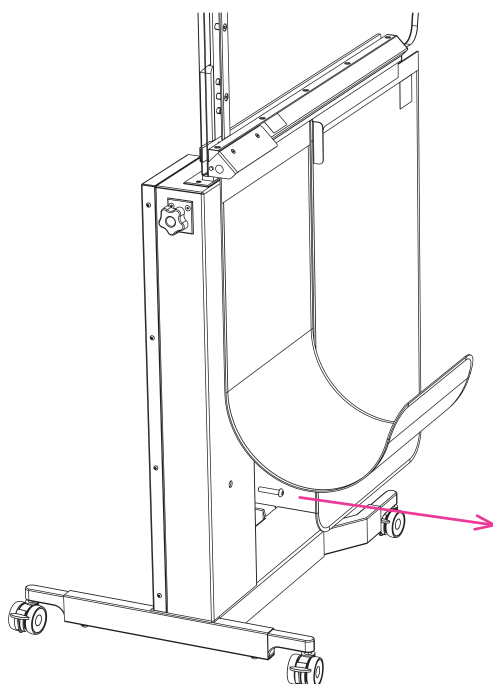
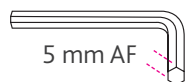
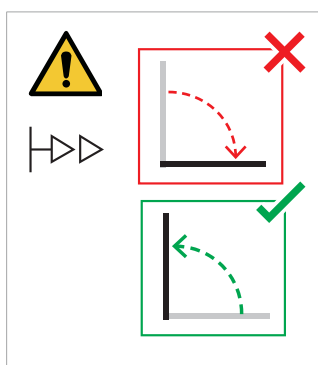
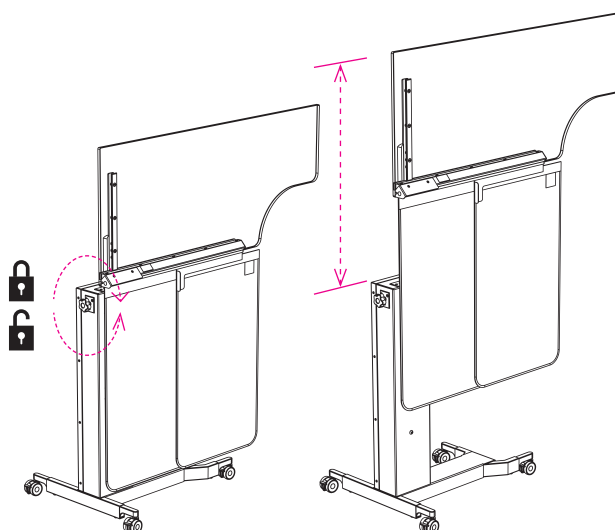
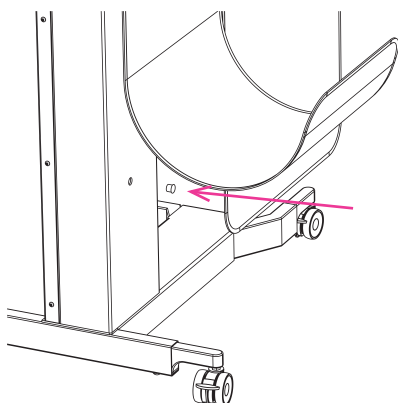
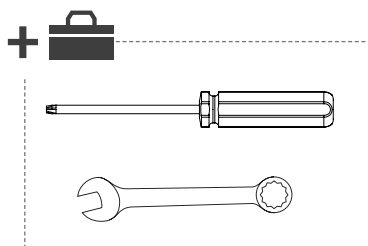
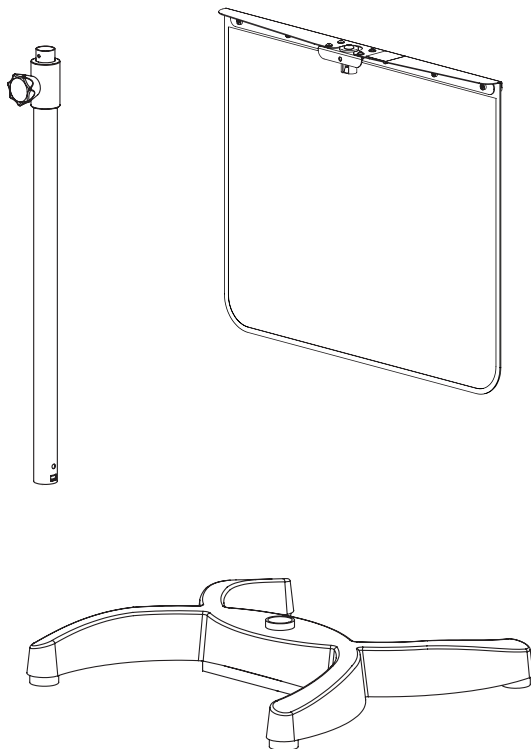
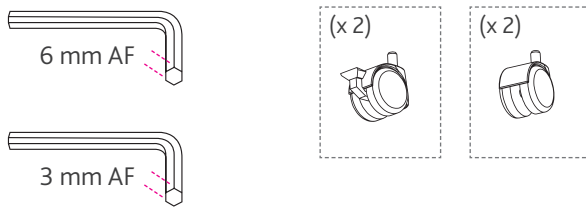
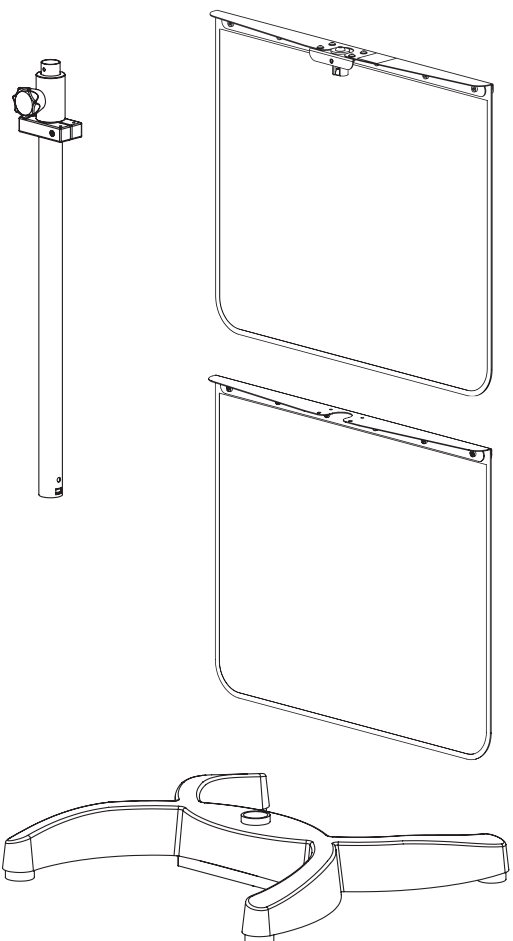
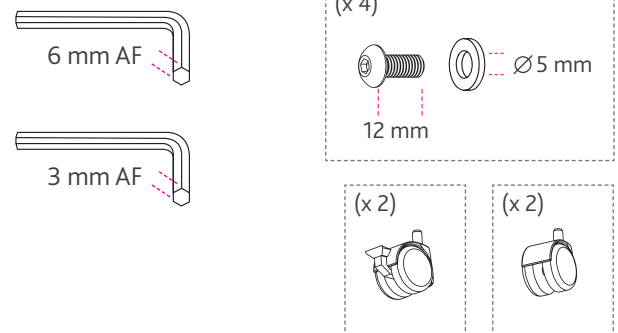
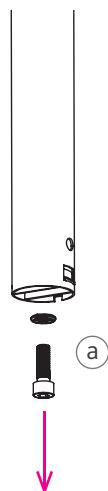
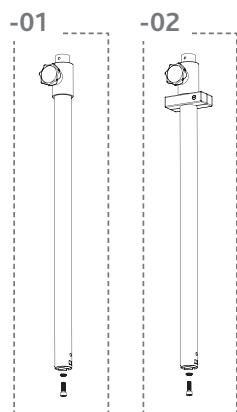
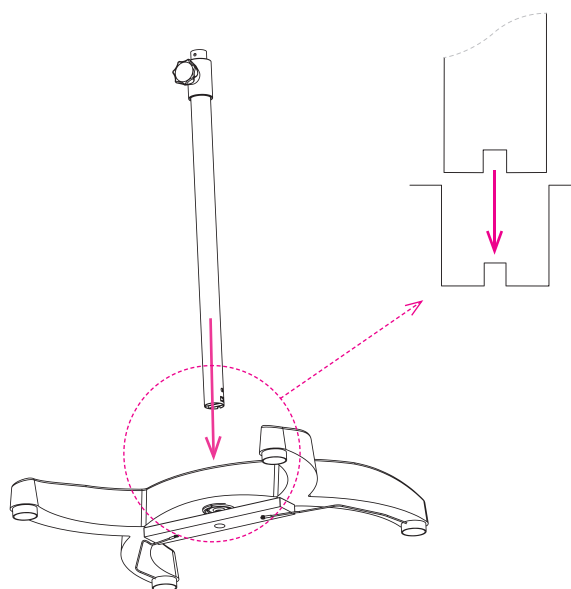
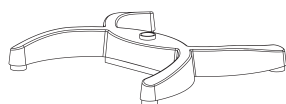
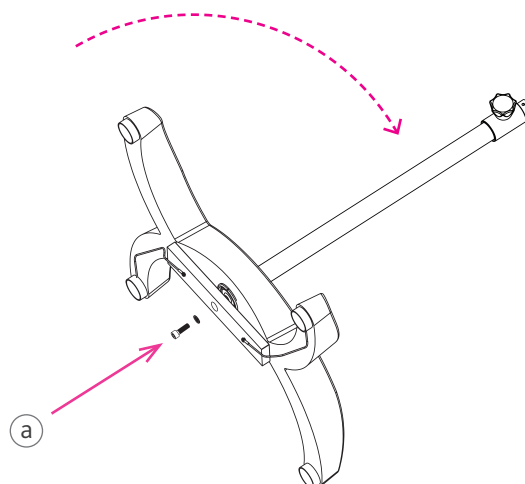
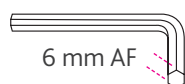
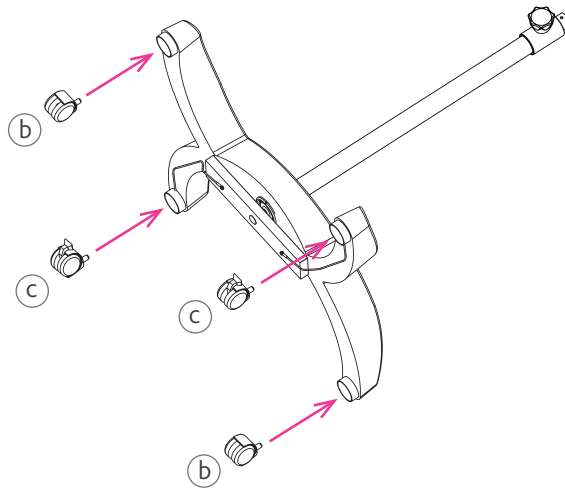
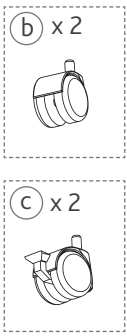
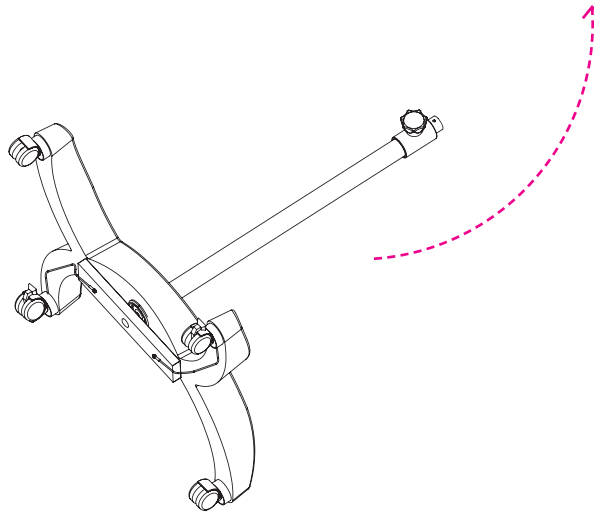
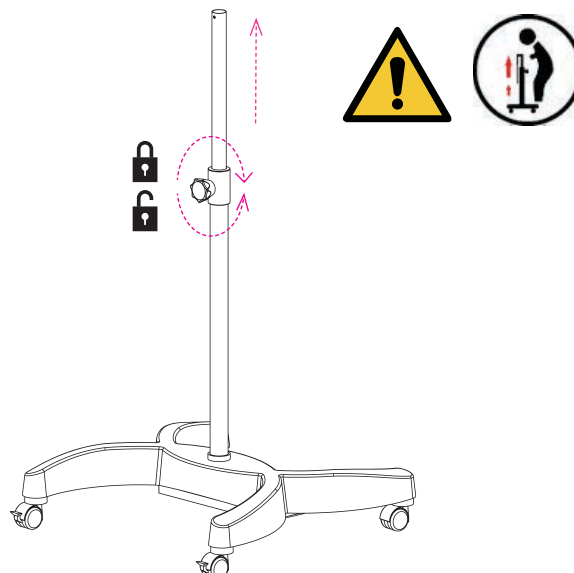
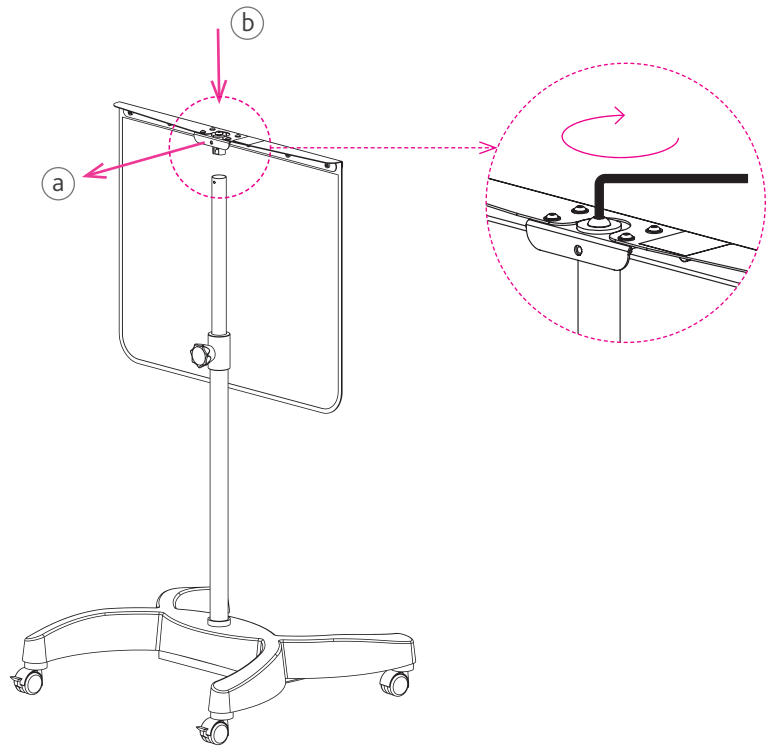
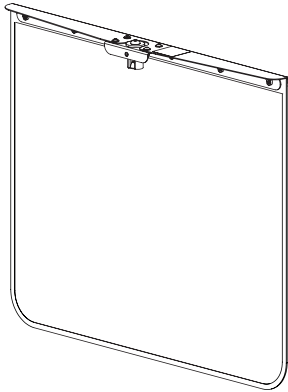
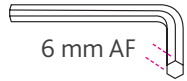
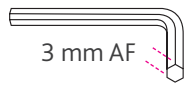
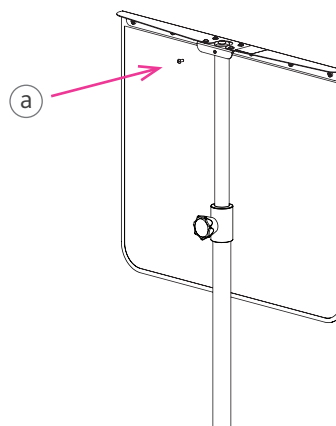
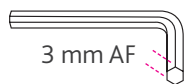
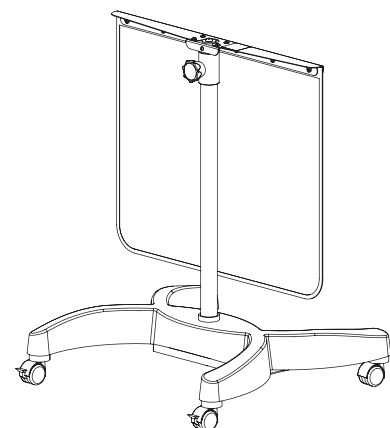
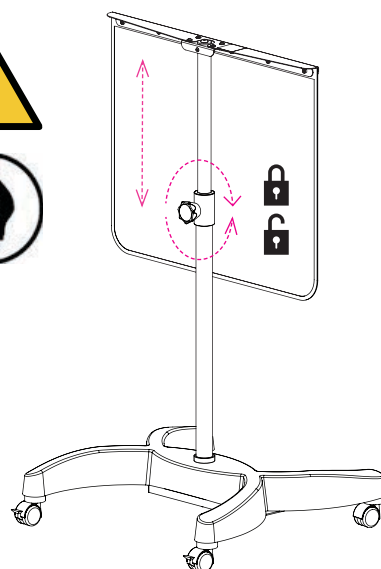
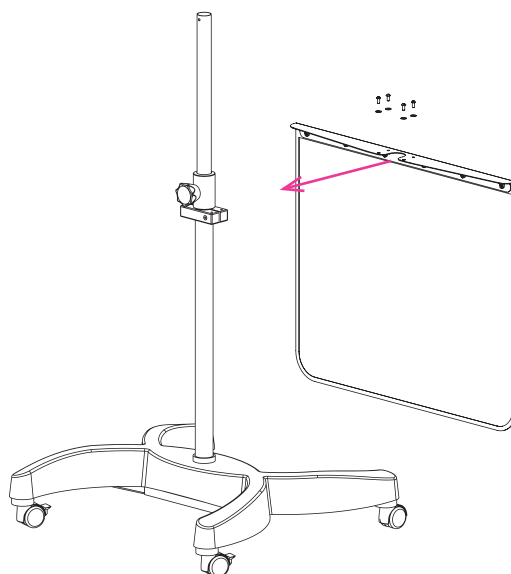
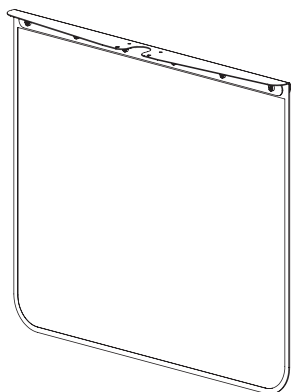
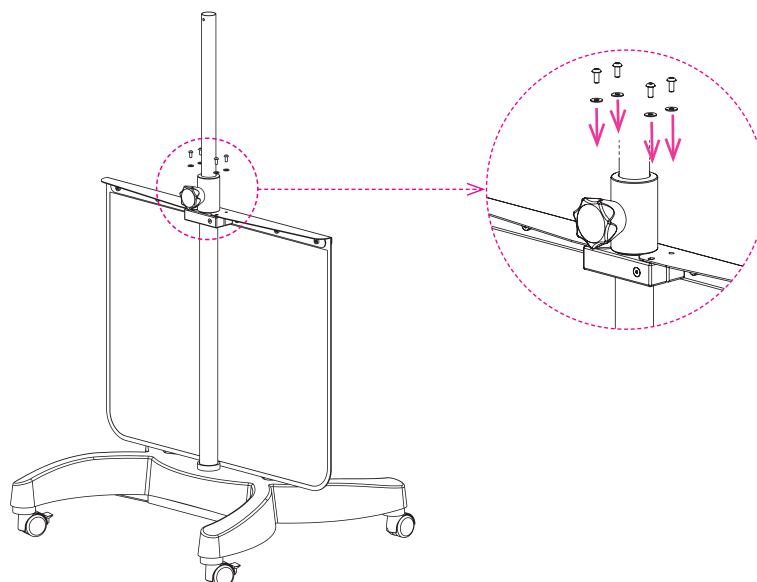
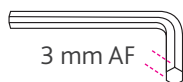
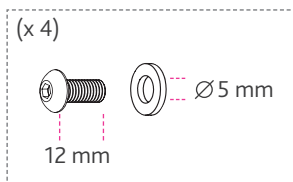
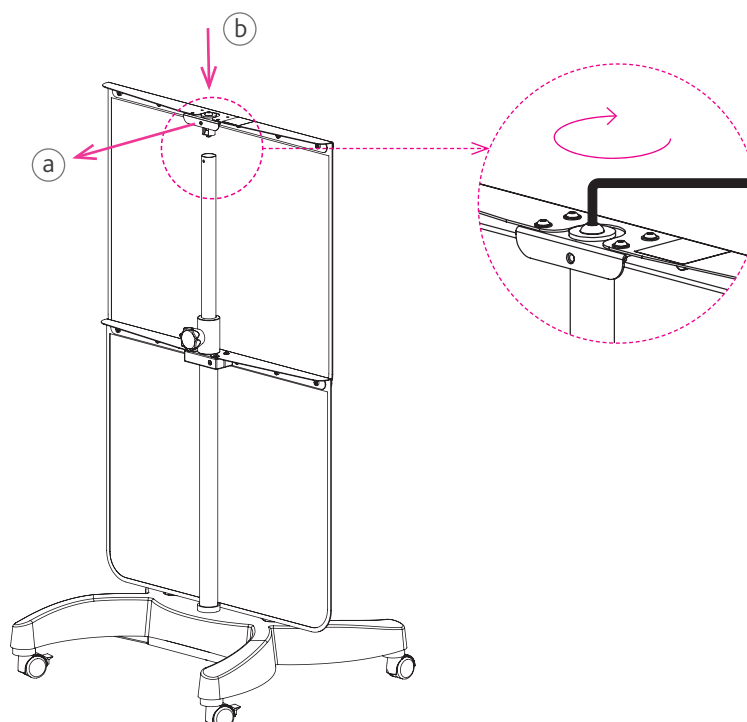
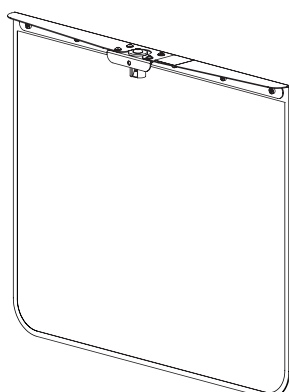
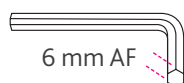
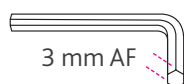
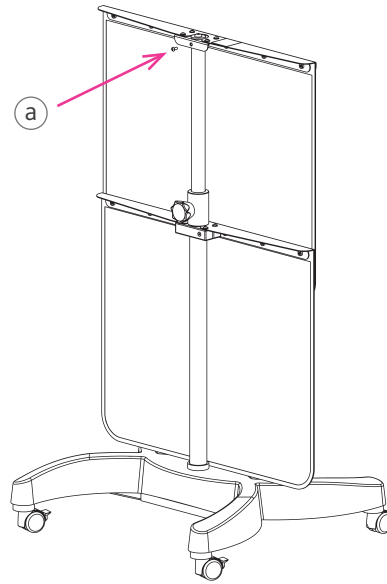
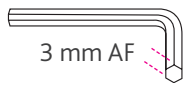
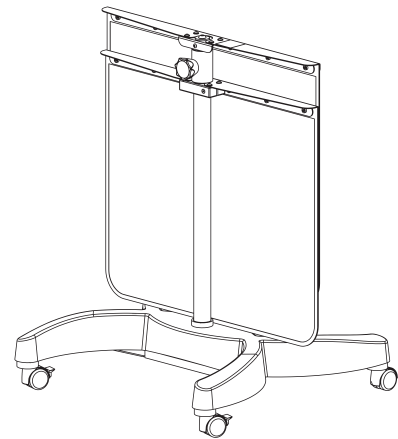
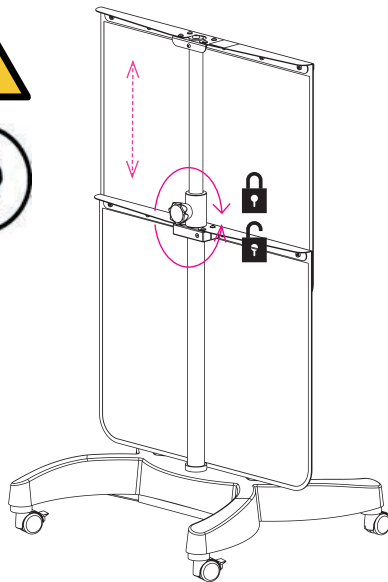
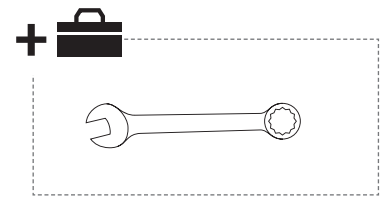
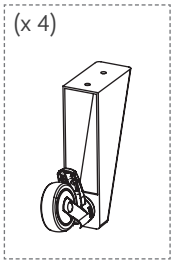
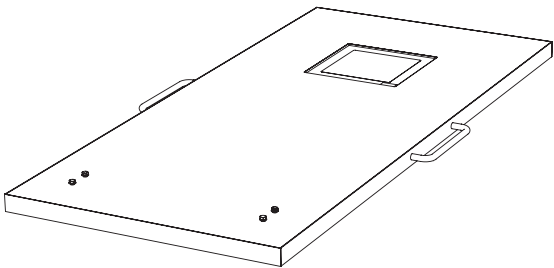
1

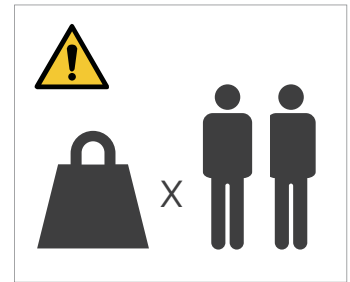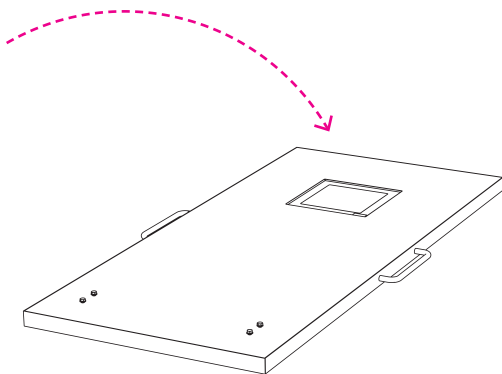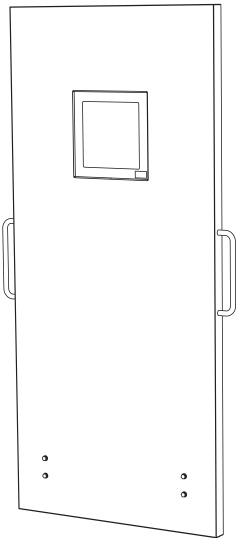2
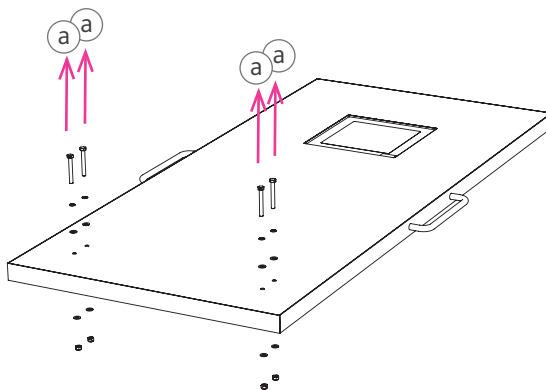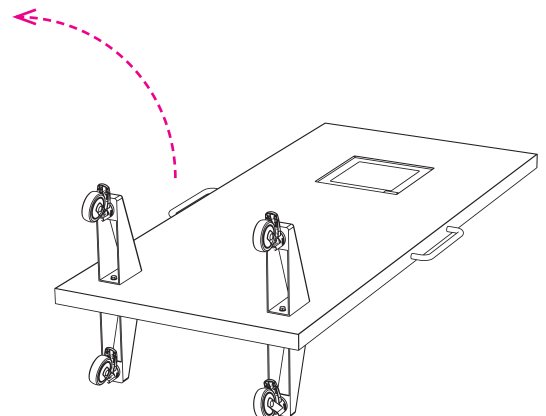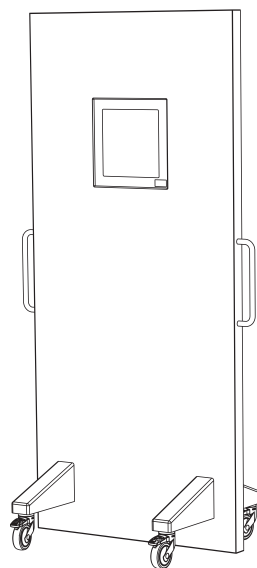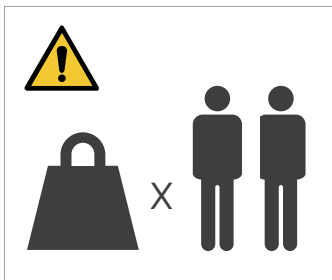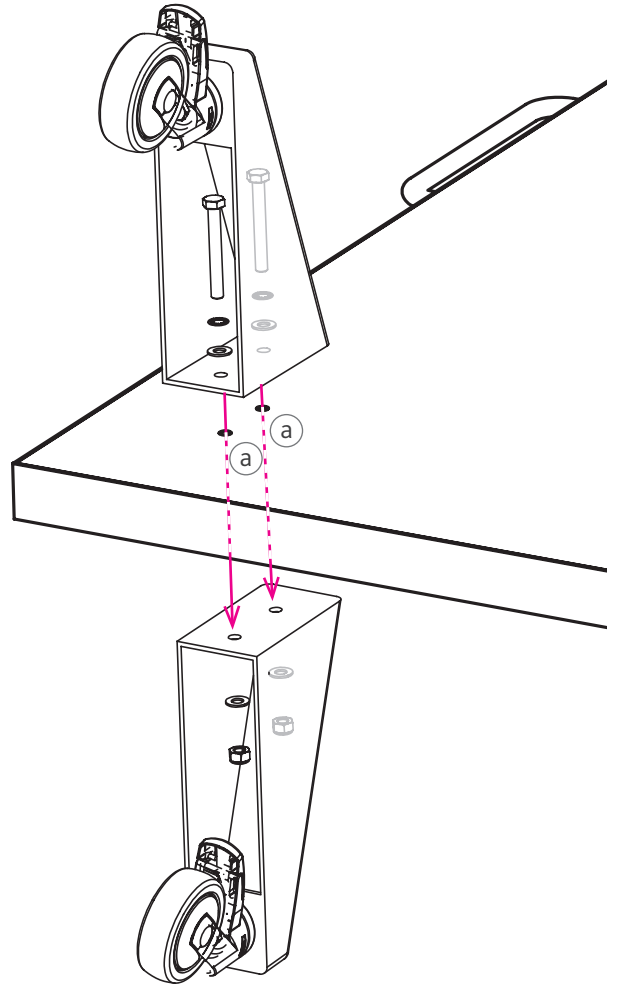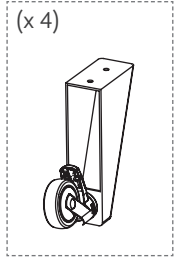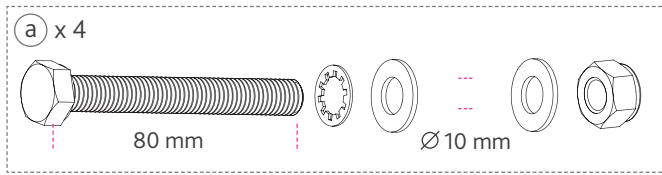
### Quality management systems and certification

Kenex is compliant and certified according to the following regulations and standards regarding design and manufacture of medical devices: -

2017/745 Medical device regulation (MDR)  
2016/425 Personal protective equipment (PPE)  
BS EN ISO 13485:2016 (Quality management systems for medical devices)  
BS EN 61331-1:2014 (Determination of x-ray attenuation properties)  
BS EN 61331-2:2014 (Translucent protective plates)  
BS EN 61331-3:2014 (X-ray protective clothing)  
BS EN 60601-1:2021 (Medical electrical equipment – basic safety and essential performance)

Quality management systems audited by SGS UK Limited.

#### EU authorised representative:

Emergo Europe, Westervoortsedijk 60, 6827 AT Arnhem,  
The Netherlands.

Emergo Single Registration Number (SRN): NL-AR-000000116

Kenex Single Registration Number (SRN): GB-MF-000009022

#### Swiss authorised representative:

MedEnvoy Switzerland, Gotthardstrasse 28, 6302 Zug, Switzerland.

#### Australian Sponsor

Emergo Australia, Level 20, Tower II, Darling Park,  
201 Sussex Street, Sydney, NSW 2000, Australia.

#### US FDA AGENT

Emergo Global Representation LLC, 2500 Bee Cave Road, Building I,  
Suite 300, Austin, Texas 78746, USA.

Kenex (Electro-Medical) Limited  
Unit 17, RO24 Greenway  
Harlow Business Park  
Harlow, Essex  
CM19 5QB  
England

T: +44 (0)1279 417241  
F: +44 (0)1279 443749  
E: [kenex@kenex.co.uk](mailto:kenex@kenex.co.uk)  
[www.kenex.co.uk](http://www.kenex.co.uk)