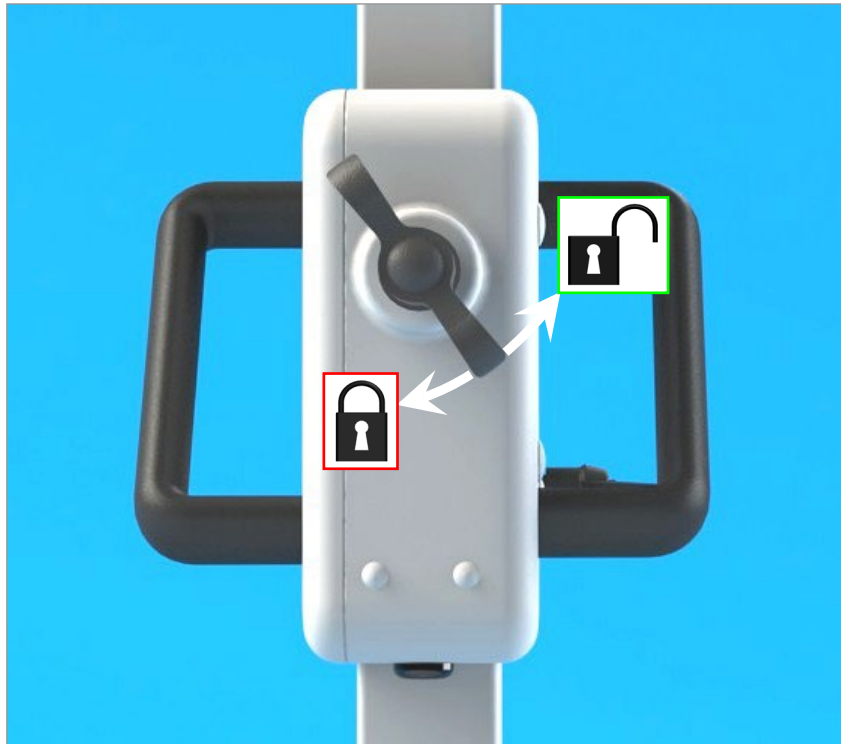
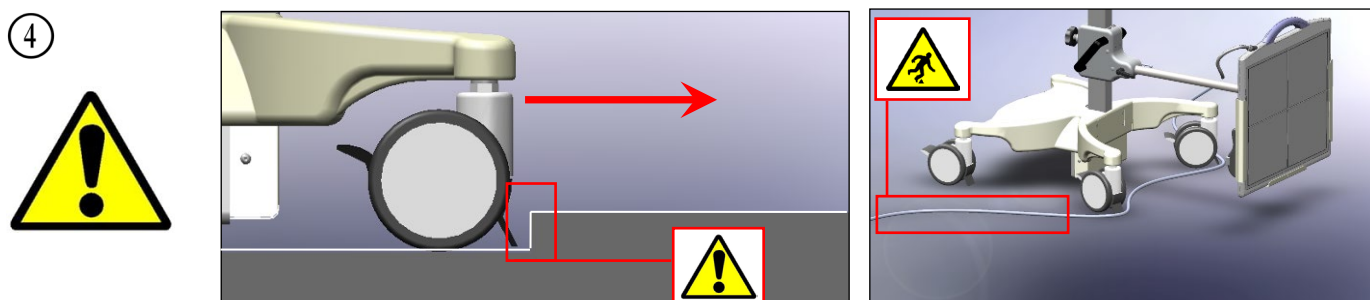
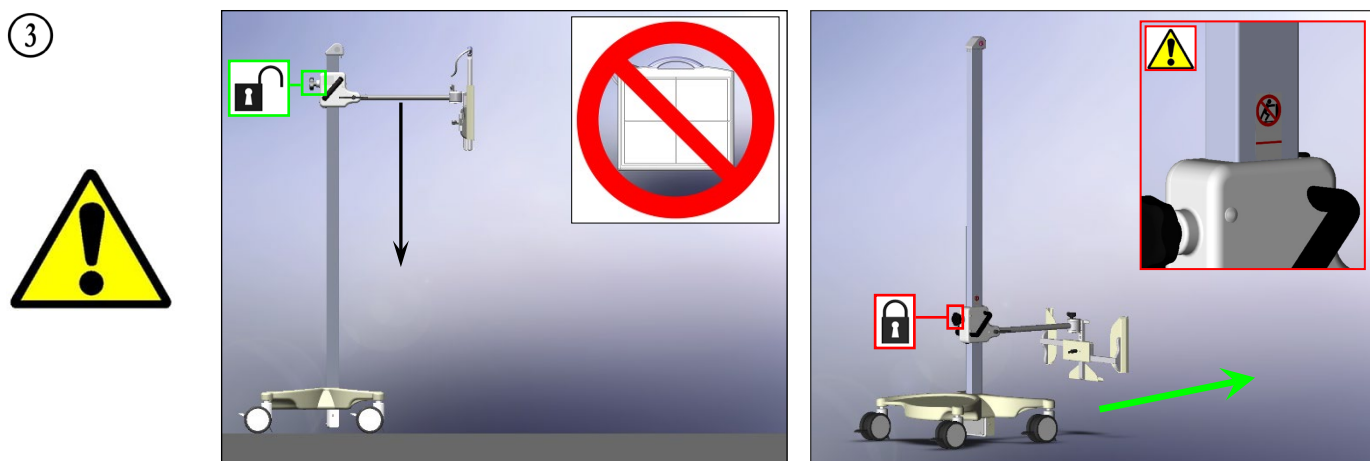
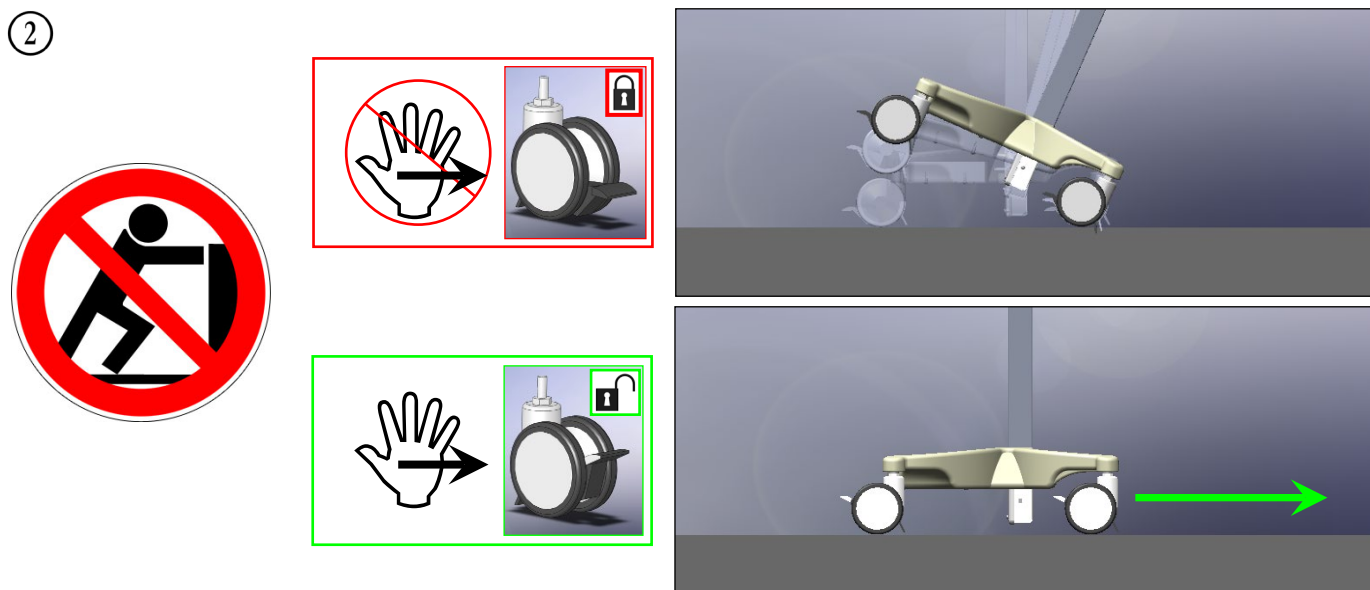
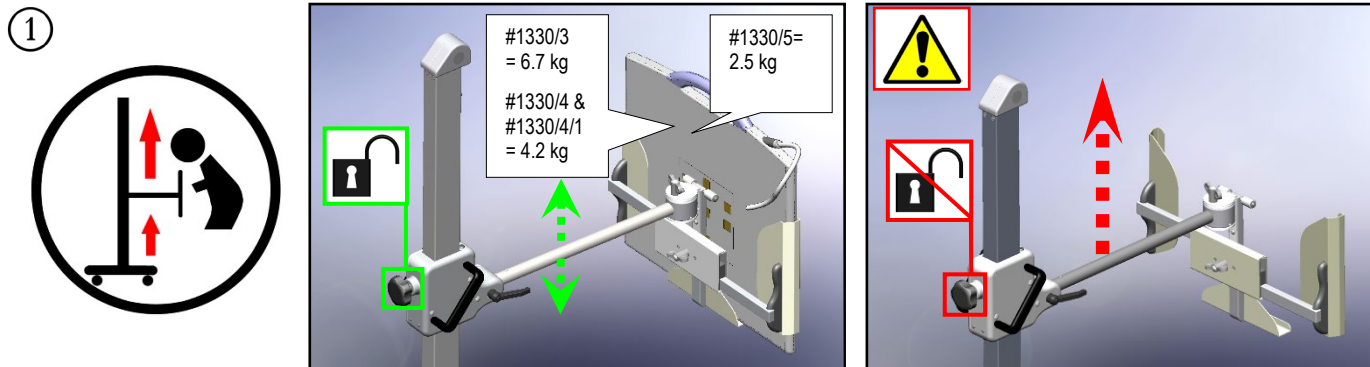
User Instructions

Mobile Detector Holder


Model: Kenex 1330/4/1 (GE E06731BL)



5

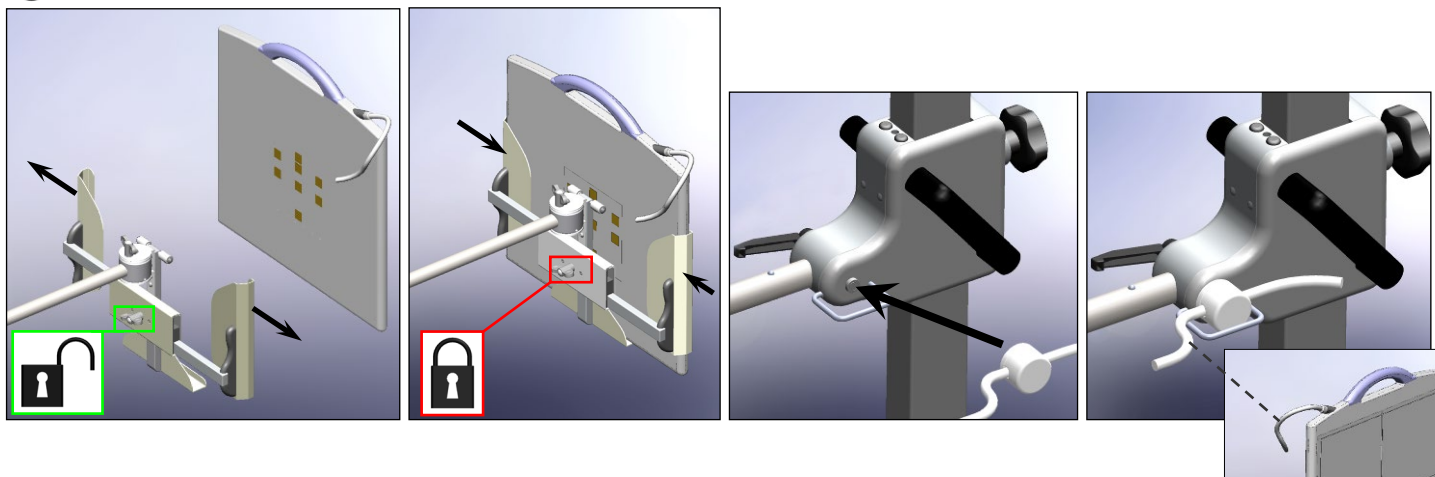6

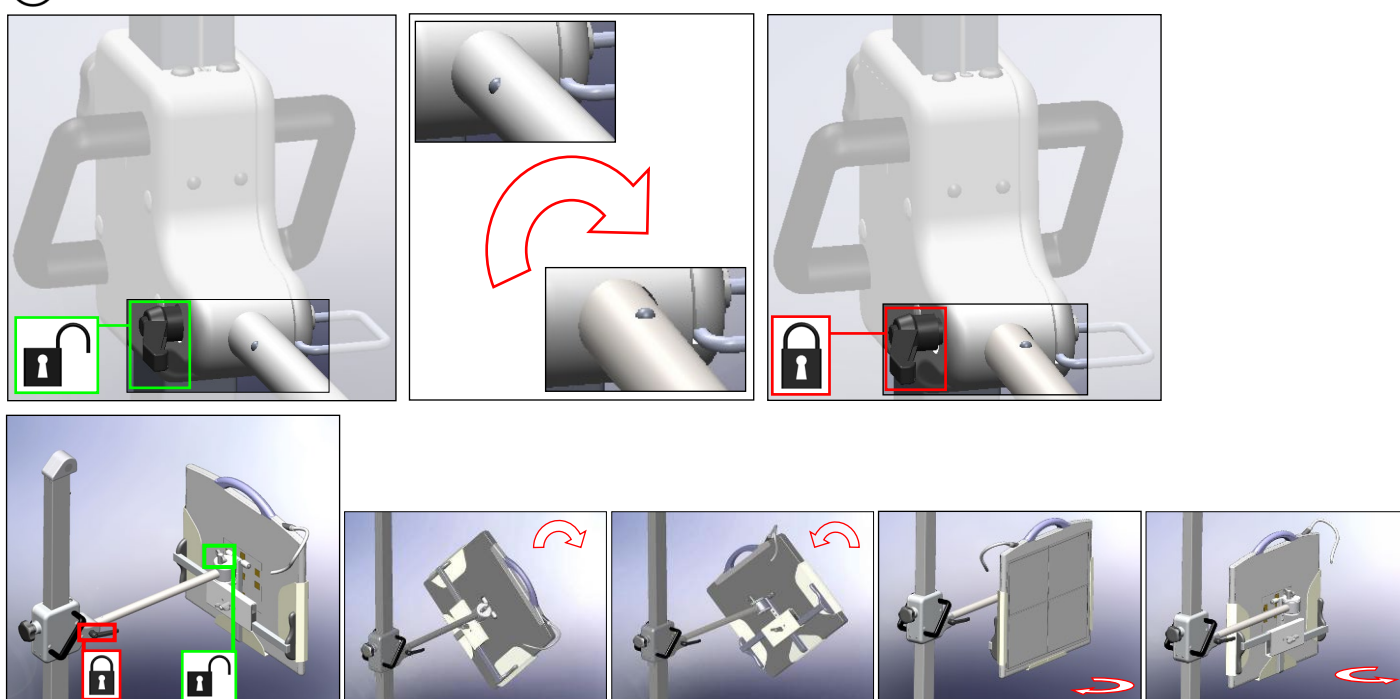7

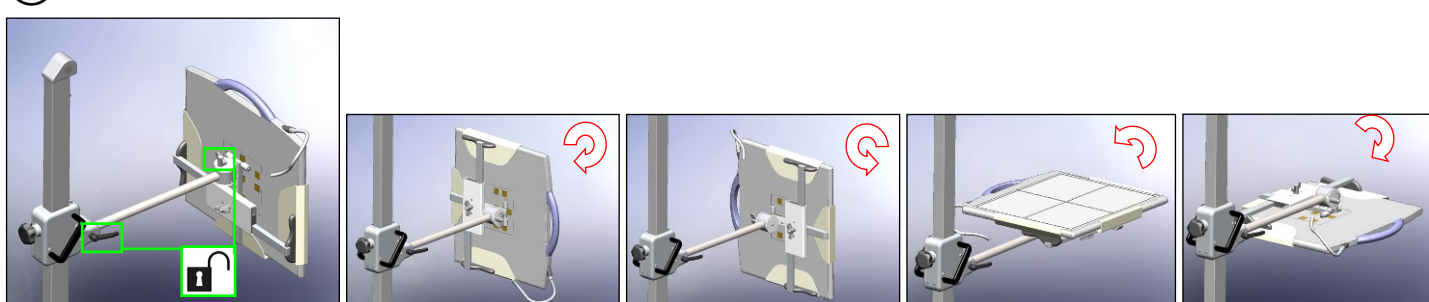DE	 7	SK	 14
GB	 7	EE	 15
FR	 8	LV	 15
ES	 8	LT	 16
SE	 9	PL	 16
IT	 9	NO	 17
RU	 10	BG	 17
DK	 10	RO	 18
GR	 11	SI	 18
CN	 11	TR	 19
FI	 12	JP	 19
PT	 12	HR	 20
NL	 13	ID	 20
HU	 13	VN	 21
CZ	 14	KR	 21

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Scope of use

This mobile holder is designed for use with a portable digital detector (maximum weight 6.7kg #1330/3, 4.2kg #1330/4 & 1330/4/1, 2.5kg #1330/5, with or without grid), with sizes up to 53.5cm wide x 3.3cm thick. It enables the detector to face all directions and is height adjustable from floor level to 120cm above floor level. The holder will accommodate CR or film cassettes of all widths from 24cm to 55cm.

Cleaning

All devices should be cleaned using a mild soap solution or a regular hand dishwashing product. Do not spray on – wipe on with a damp cloth.

Any serious incident that occurs in relation to these devices should be reported to Kenex and to the local competent authority for medical devices.

①

CAUTION: The holder will move up rapidly (especially from a low level) if the detector is removed and the lobe knob unlocked. Always hold the horizontal extension pole or handles to safely control vertical movement.

②

CAUTION: Do not lean against or push the equipment with the brakes activated as it may overbalance.

③

CAUTION: Ensure the lower edge of the holder is positioned just above the floor for transportation purposes.

④

CAUTION: Each castor is fitted with a cable guard that prevents the detector holder from running over and damaging an x-ray interface cable as the base moves forward. These guards can therefore strike a small step or floor protrusions. Care should be taken when moving the equipment.

CAUTION: Care must be taken to avoid tripping over the cable.

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EU authorised representative:

Emergo Europe, Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands.

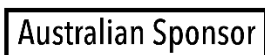
Emergo Single Registration Number (SRN): NL-AR-000000116

Kenex Single Registration Number (SRN): GB-MF-000009022

Swiss authorised representative:

MedEnvoy Switzerland, Gotthardstrasse 28, 6302 Zug, Switzerland.

EMERGO Asian-Pacific Pty. Ltd., 201 Sussex Street, Level 20, Tower II, Darling Park, Sydney, NSW 2000, Australia.

EMERGO Global Representation LLC, 2500 Bee Cave Road, Building I, Suite 300, Austin, Texas 78746, USA.

Kenex (Electro-Medical) Ltd
Unit 17, RO24 Greenway, Harlow Business Park, Harlow, Essex CM19 5QB, England
T: +44 (0)1279 417241 F: +44 (0)1279 443749 E: kenex@kenex.co.uk www.kenex.co.uk