



Installation instructions

Head end shield


Model 313/A2-001 (Shield & wall rack for use with a 525 mm wide Artis table or 520 mm wide TruSystem 7500 for Artis zee systems)

Shield and wall rack

The width of the shield and wall rack must be set dependent upon which table the shield is to be used on. Follow the steps below before using the shield.

Step 1

Using a 4 mm a/f Allen key, loosen screw (A) before adjusting the captive screw (B) to alter the width of the shield allowing it to slide onto the desired table (turn clockwise to reduce width). Ensure that good grip onto the table is maintained, along with ease of shield movement. Tighten screw (A).

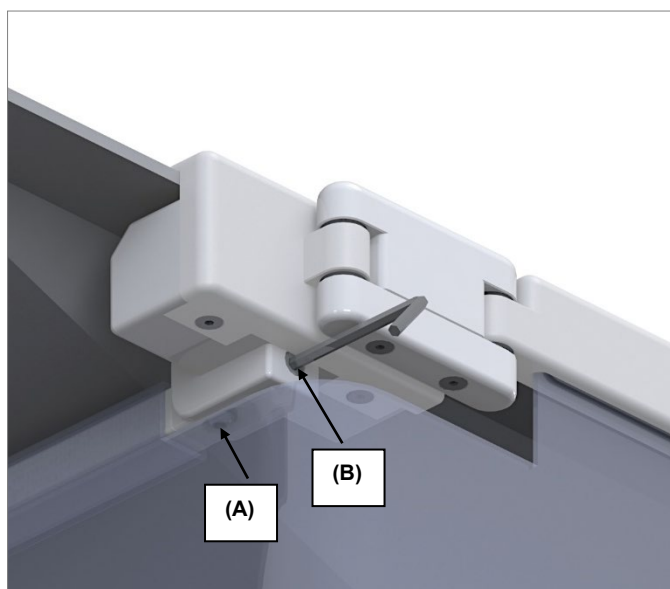

Step 2

Using a 3 mm a/f Allen key, adjust the aluminium block on the wall rack according to the width of the shield.




Position the aluminium block outwards for the 525 mm wide Artis table

OR


Position the aluminium block inwards for 520 mm wide TruSystem 7500 for Artis zee systems.

Adjustable aluminium block

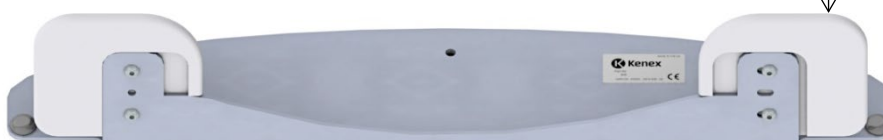



Step 3

Attach the wall rack to a suitable wall.

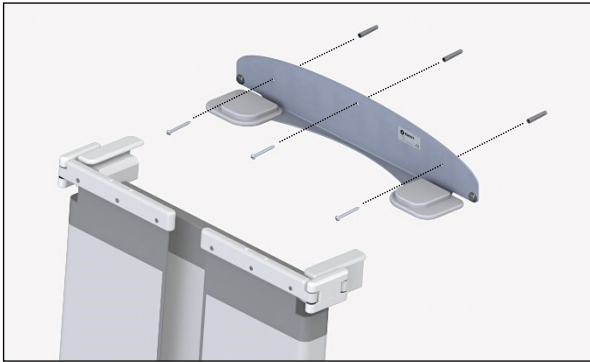





Step 4

When using the shield, ensure it is positioned correctly.

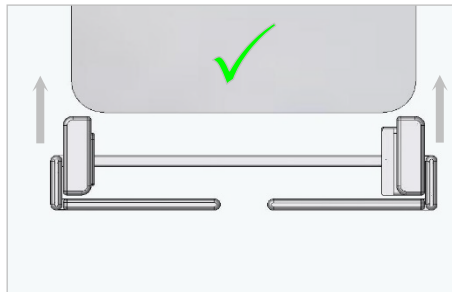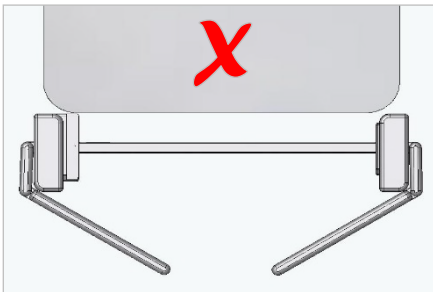

Step 5

Position shield as required.

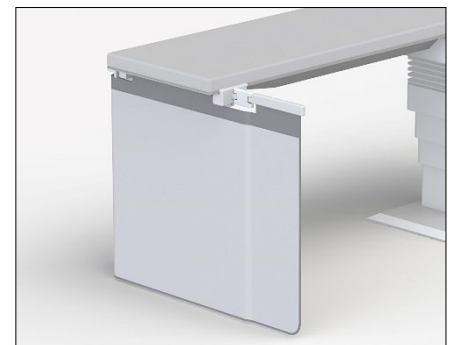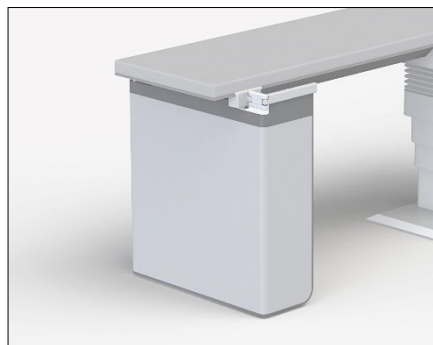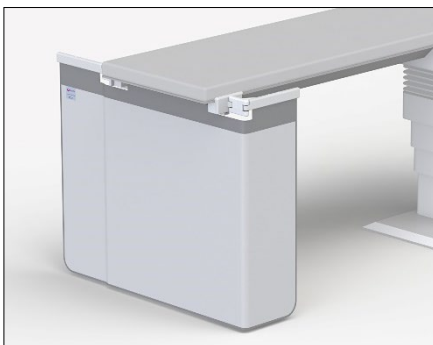
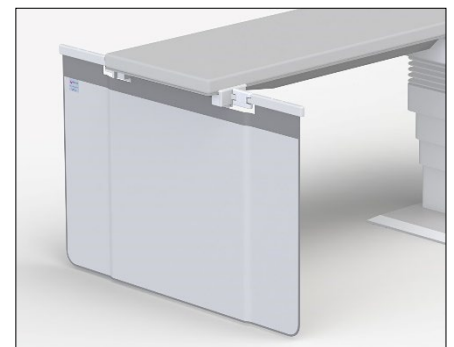


Cleaning



All devices should be cleaned using a mild soap solution or a regular hand dishwashing product. Do not spray on – wipe on with a damp cloth.

Recycling

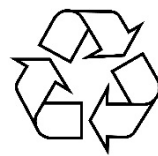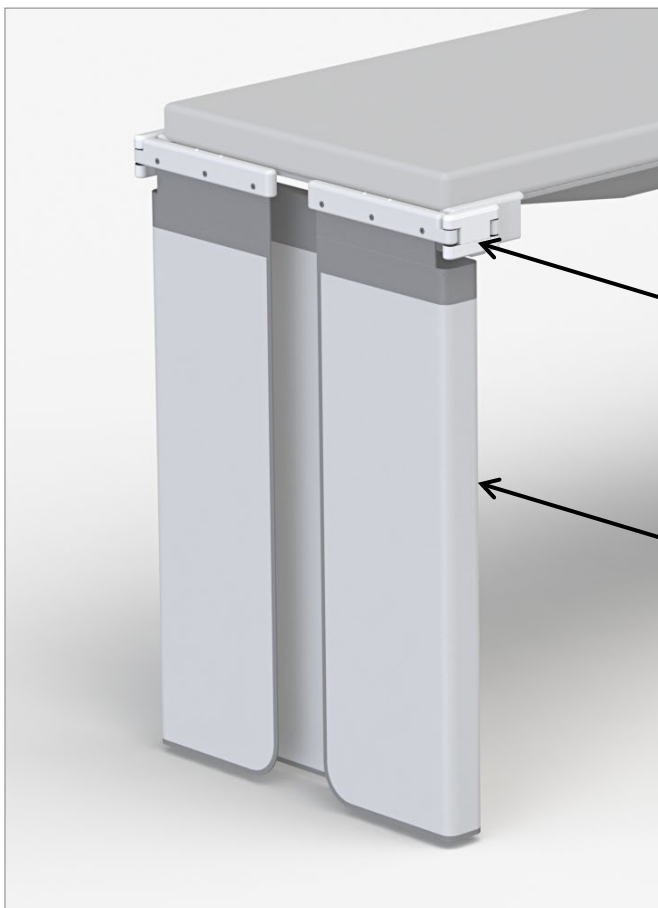

Aluminium (Al)

PVC + Pb = Classified as non hazardous waste
European Waste Catalogue (EWC) code 12 01 99