IT005

**1342/A3/E-001, 1342/A2/E-001, 1342/A2/F,
312/W-TOP, 312/W-LB, 312/W-LBT**

1342/A2/E-001

1342/A2/F

312/W-TOP

312/W-LB

1342/A3-E-001
1342/A2-E-001

4 - 19

1342/A2/F

4 - 15, 20 - 23

312/W-TOP

4 - 15, 24 - 25

312/W-LB

4 - 15, 26 - 27

312/W-LBT

4 - 15, 24 - 27

EN- Instructions for use in English

Safety symbols

WARNING

Denotes a hazard with a medium level of risk, which, if not avoided, may result in serious injury or injury requiring treatment.

CAUTION

Denotes a hazard with a low level of risk, which, if not avoided, may result in minor or moderate injury.

DO NOT OVERLOAD

Maximum load weights are stated in these instructions.

Information for safe use

Intended use

Storage solutions for radiation shields used during diagnostic X-ray procedures to protect healthcare personnel from exposure to scattered X-rays. Correct storage is important in protecting the products integrity.

Important information for use

- For use by professional personnel.
- The user must check the device is functioning correctly before use.
- Retain these instructions for future reference.
- Life expectancy 10 years with correct use and care.

Assembly

- Prior to the installation of wall mounted storage, a structural engineer of record must ensure that the wall is designed in such a way that the loads of the installed rack and the removable tops (of tableside radiation shields), can be securely borne and transferred. The load bearing capacity of the wall must be checked by structural analysis and confirmed by means of an acceptable certificate. The installation must be inspected by the engineer and certified in accordance with any regional construction regulations. The layout of the installation, the selection of appropriate installations fastenings and torque values is the sole responsibility of the structural engineer.

- Fixings are not supplied.

Routine checks

- If a device is in collision, or is not free moving, stop using it and have it inspected.

Cleaning

All devices should be cleaned using a mild soap solution or a regular hand dishwashing product. Do not spray on – wipe on with a damp cloth.

Reporting obligation

Any serious incident that occurs concerning these devices should be reported to Kenex and the local competent authority for medical devices.

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
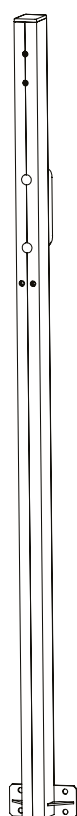
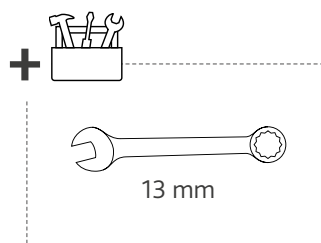
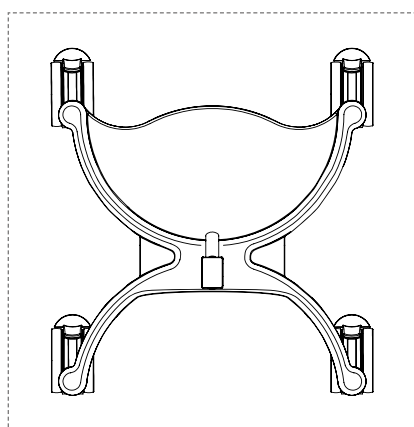
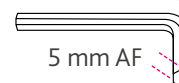
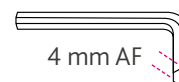
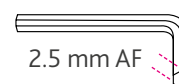
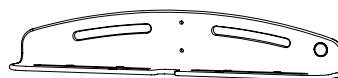
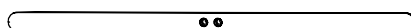
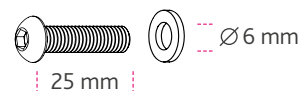
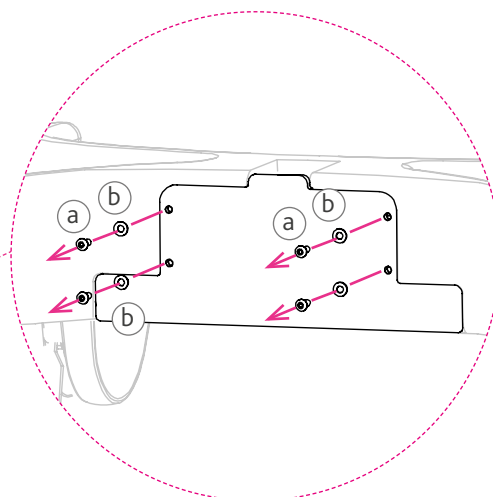
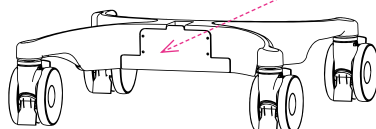
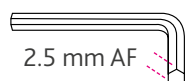
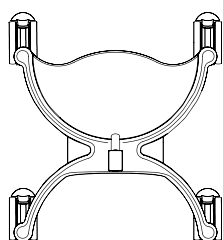
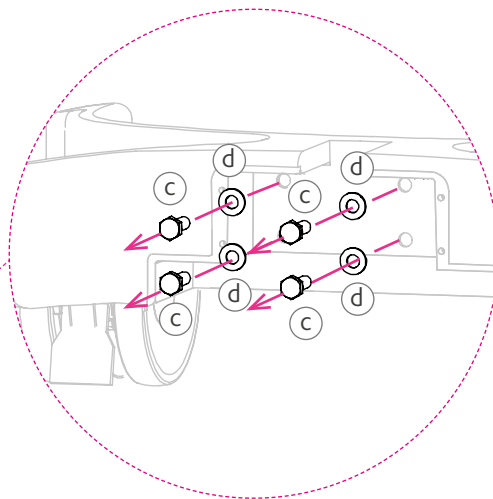
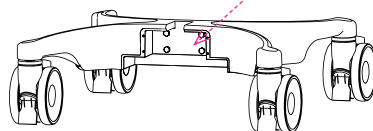
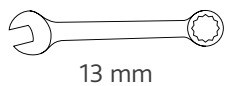
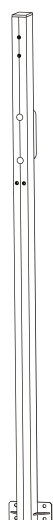
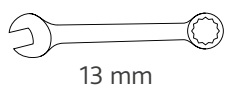
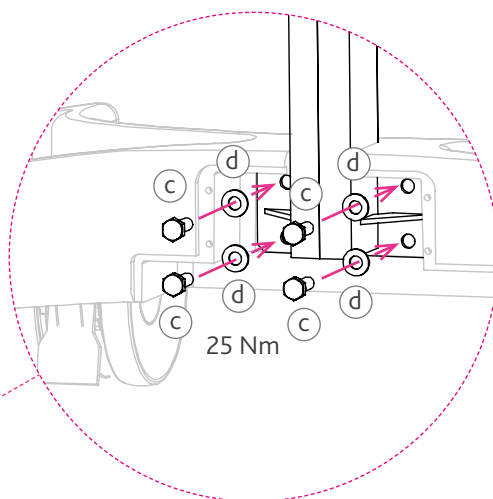
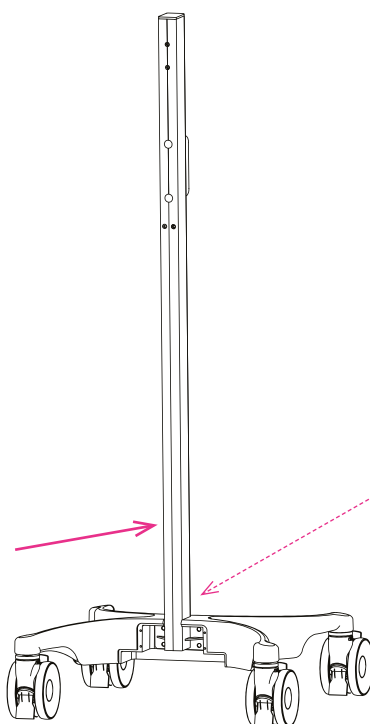
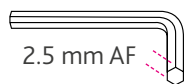
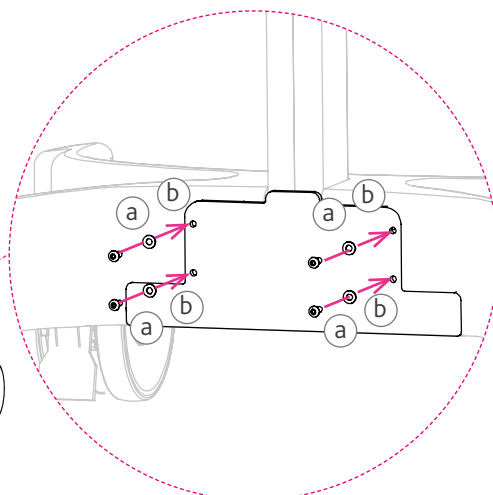
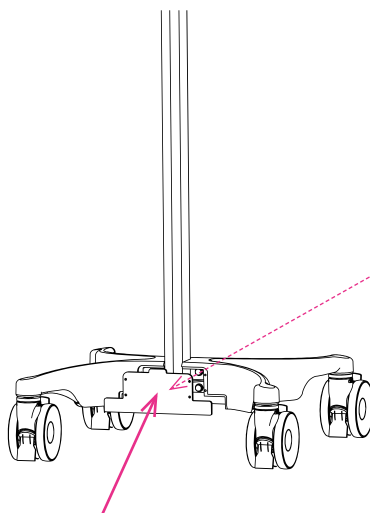
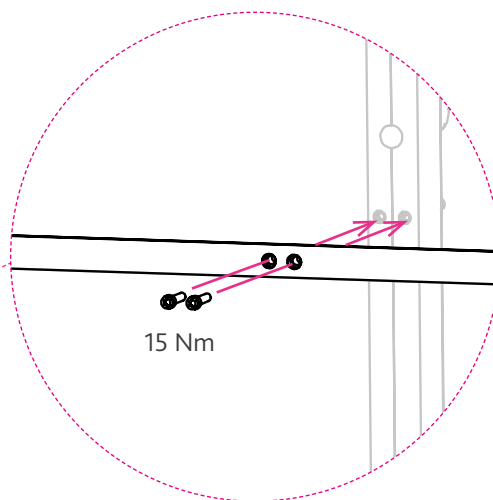
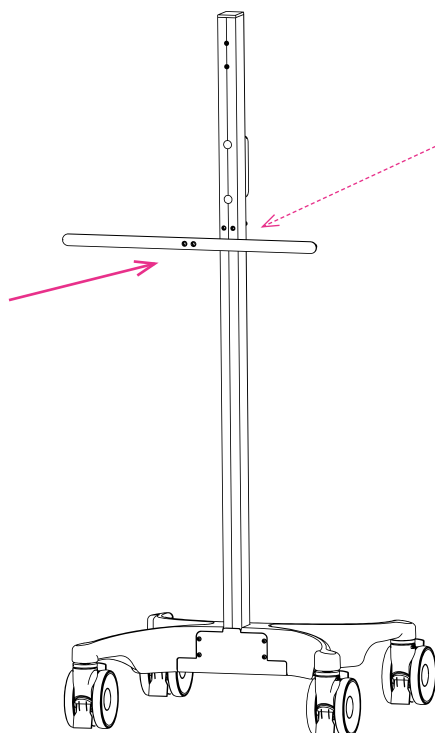
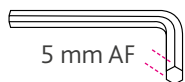
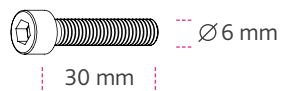
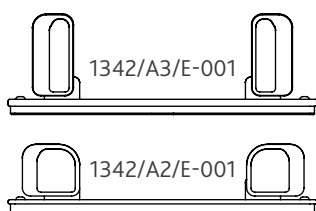
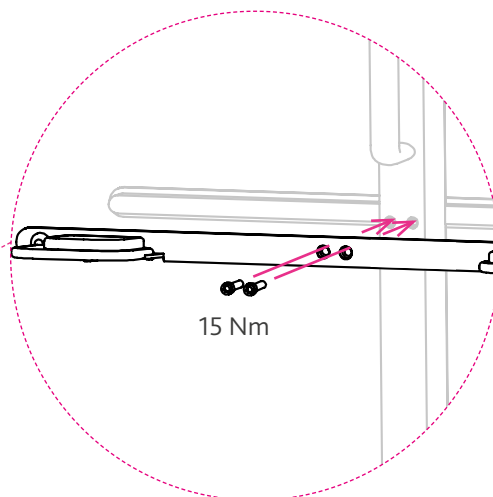
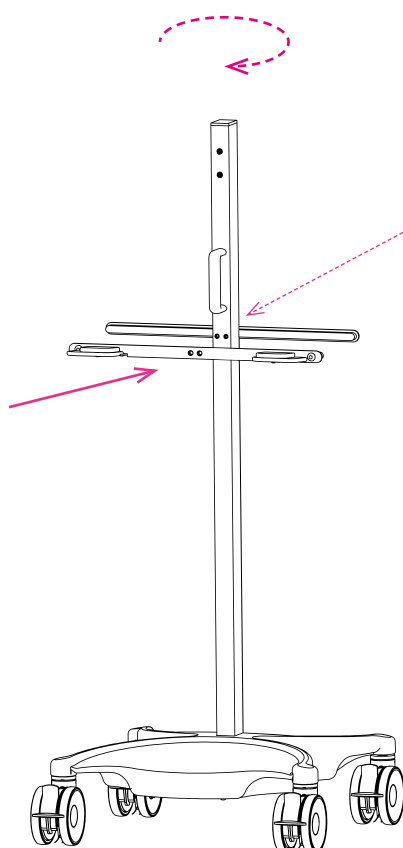
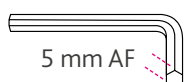
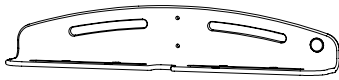
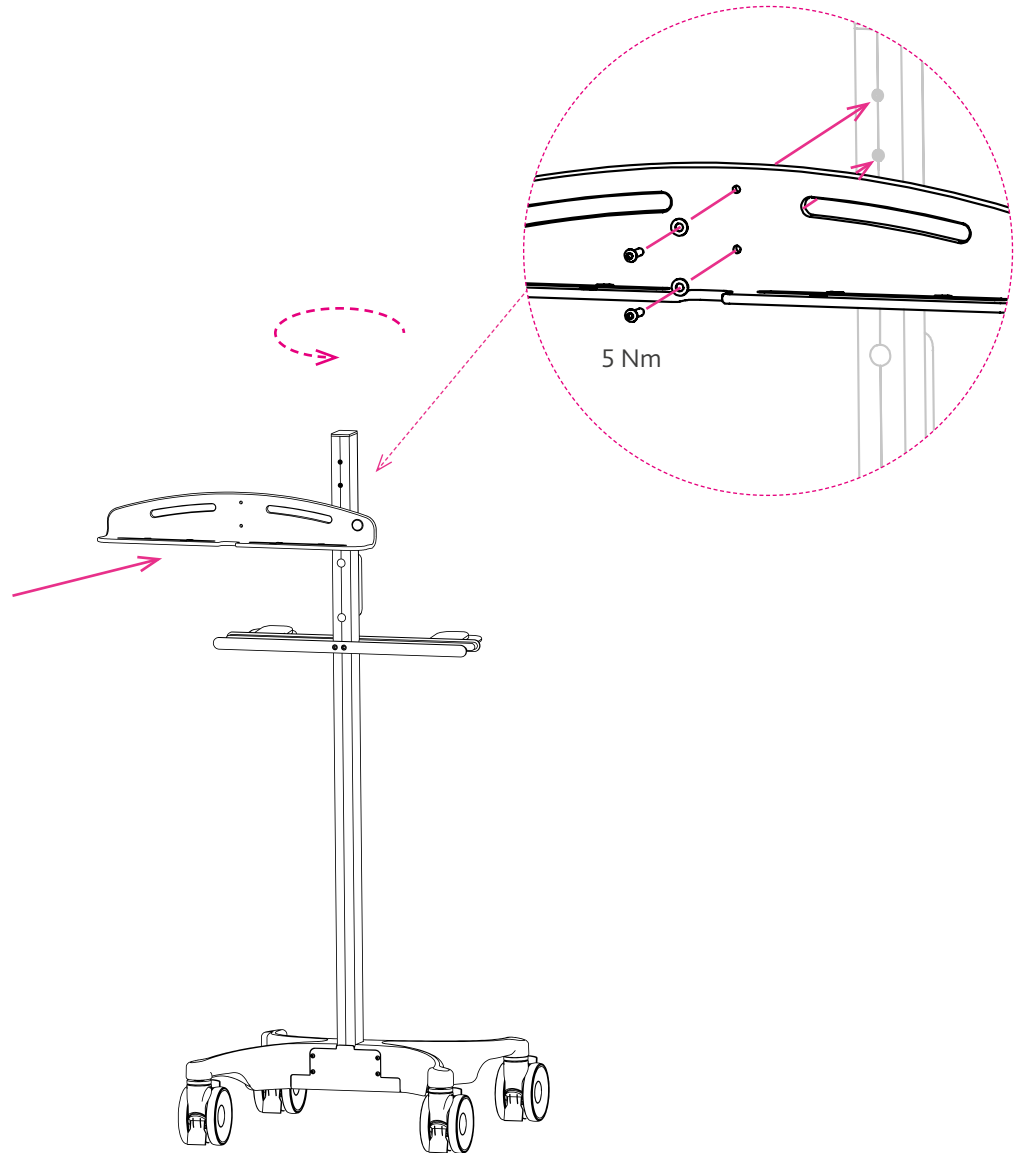
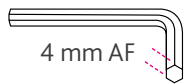
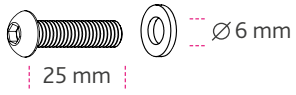
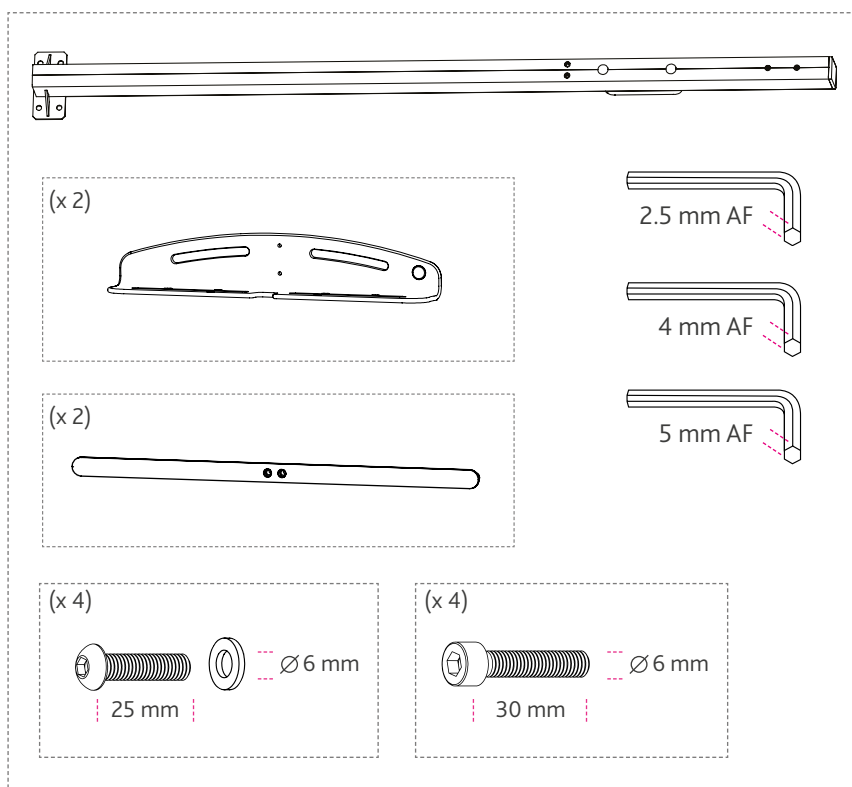
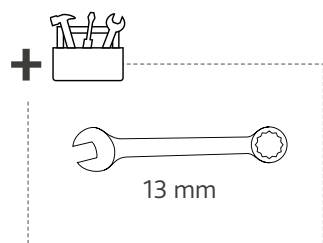
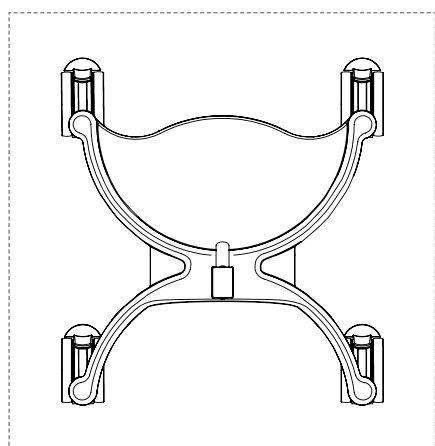
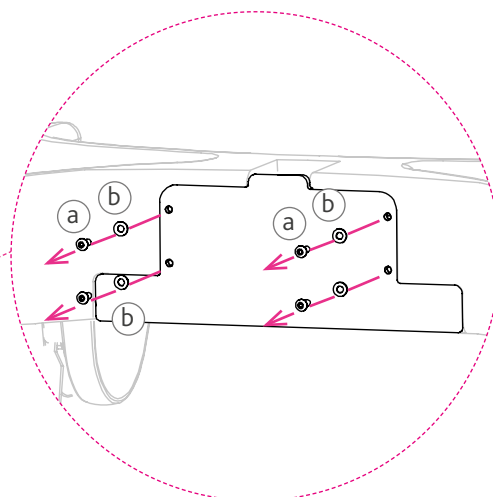
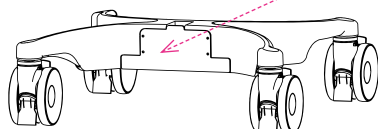
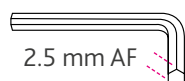
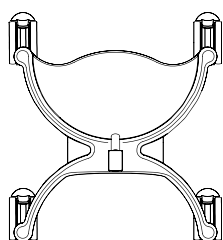
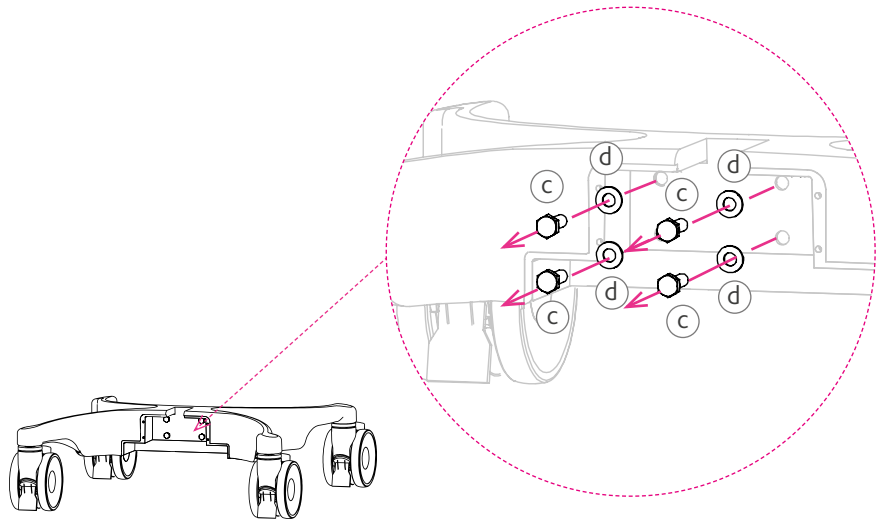
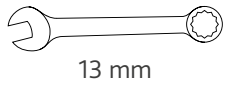
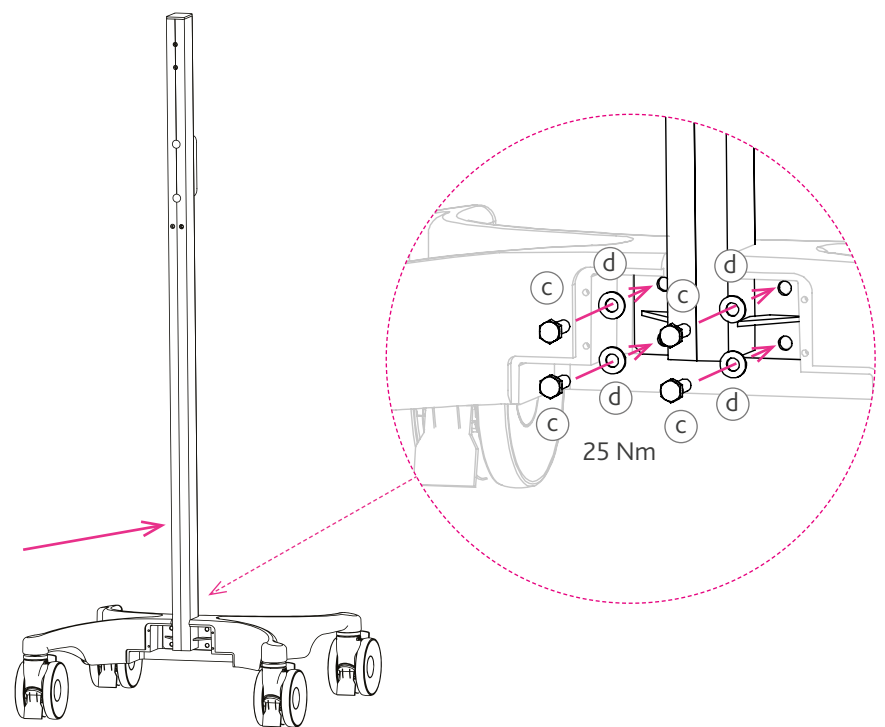
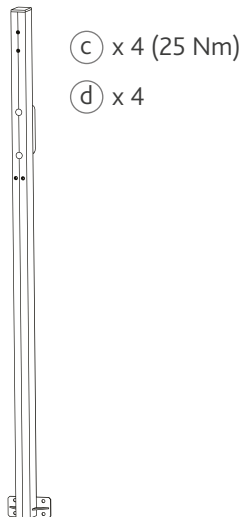
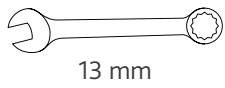
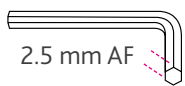
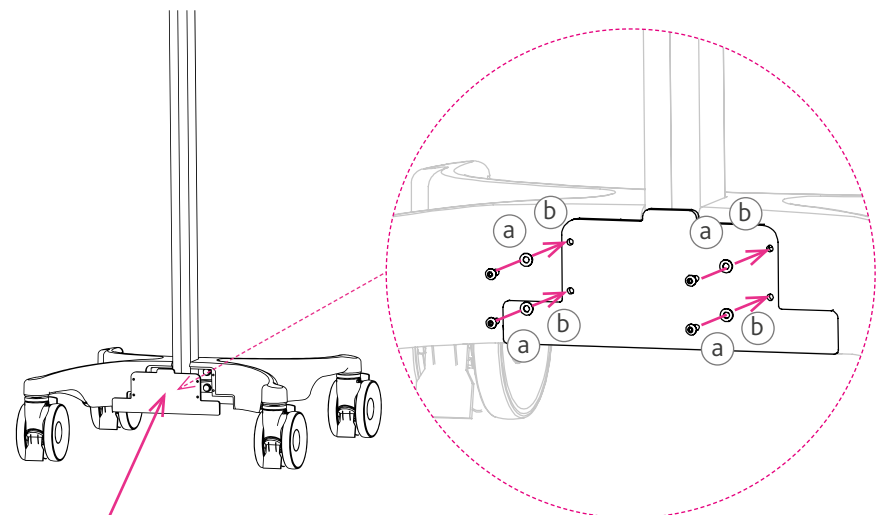
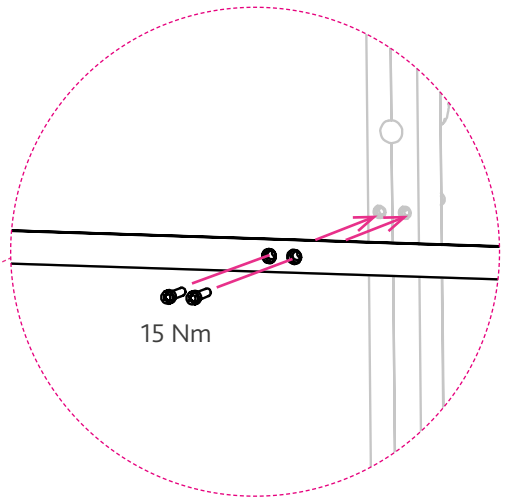
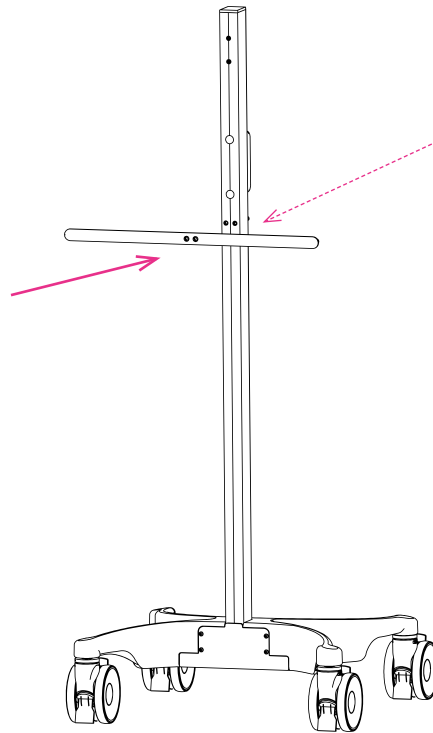
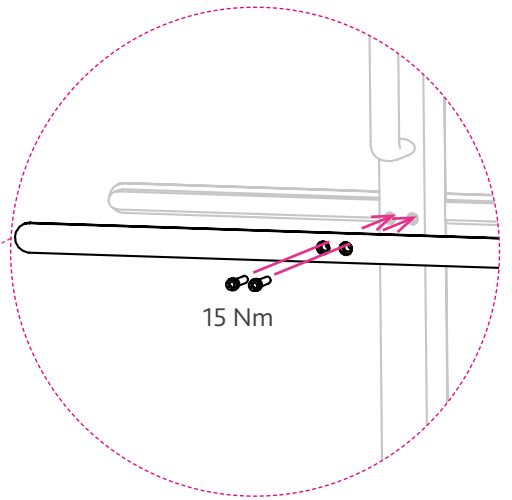
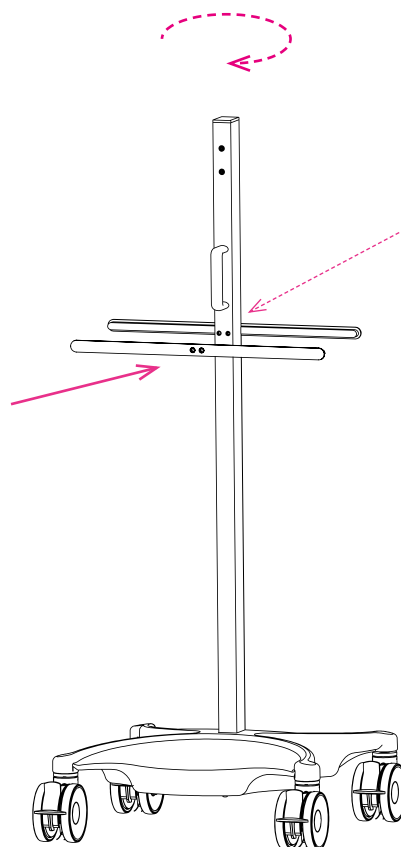
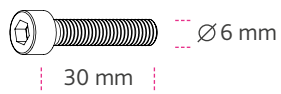
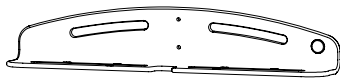
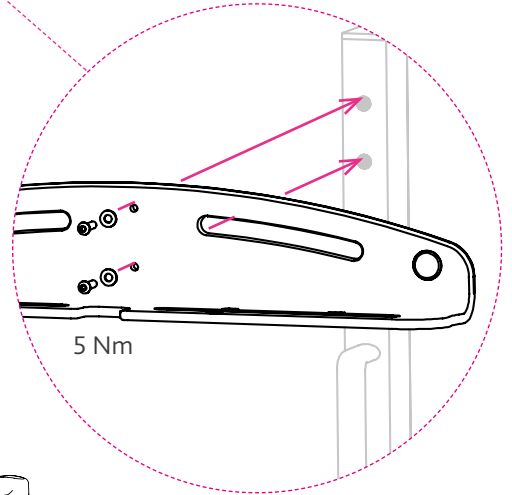
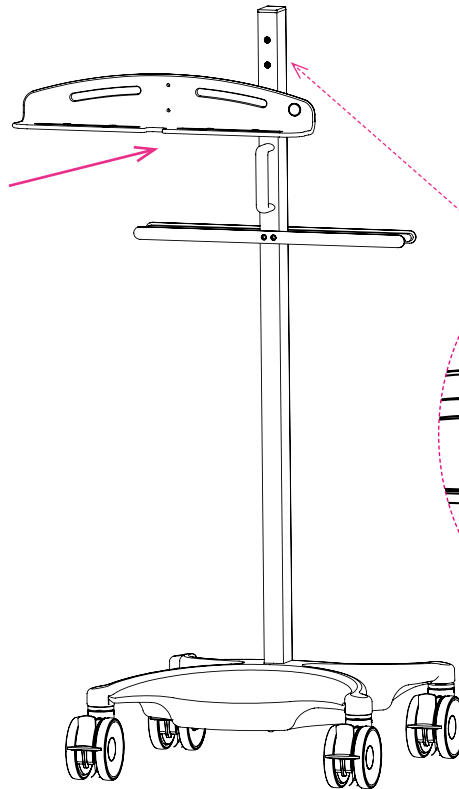
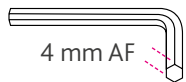
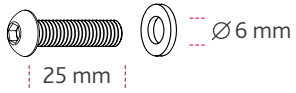
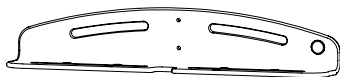
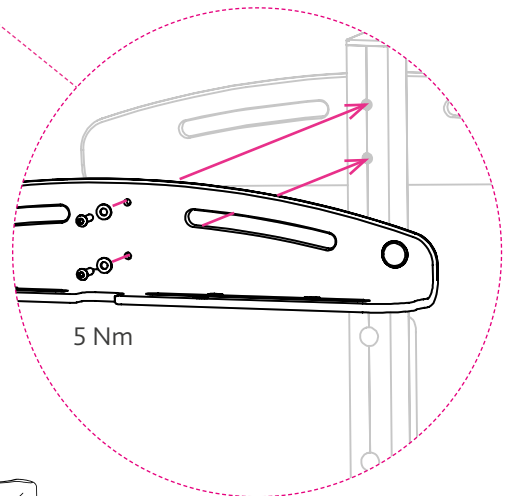
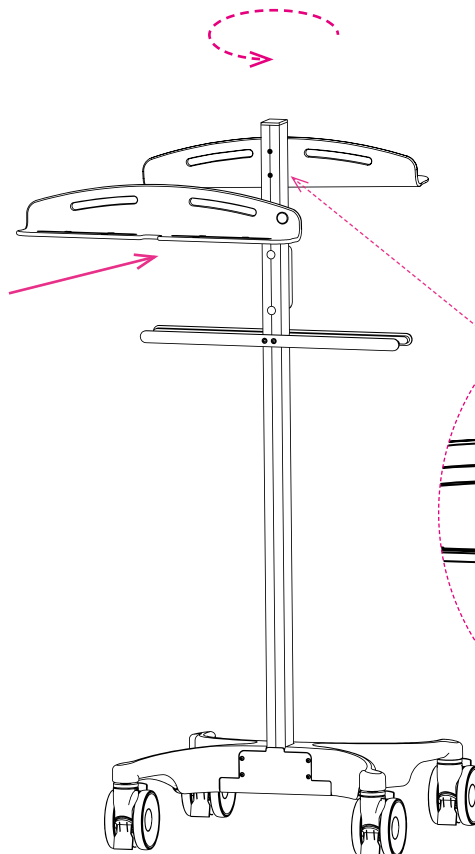
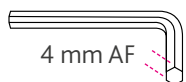
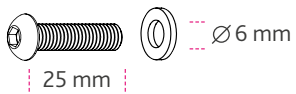
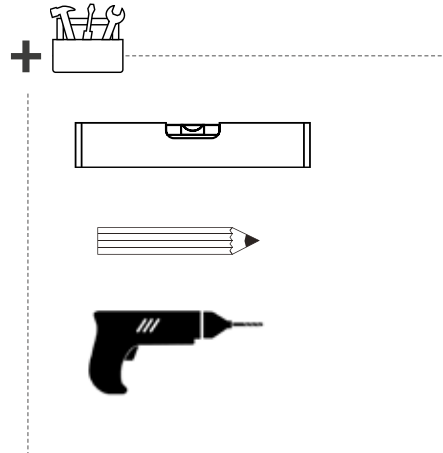
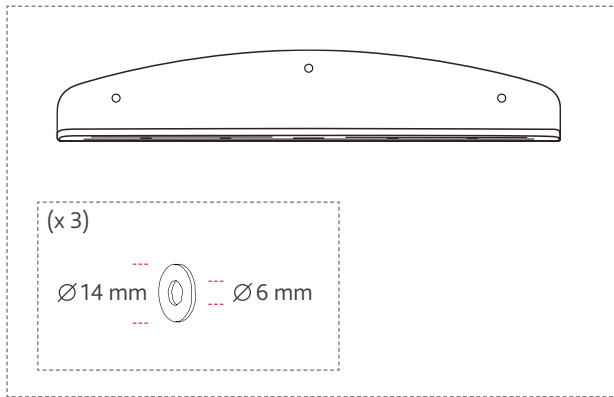
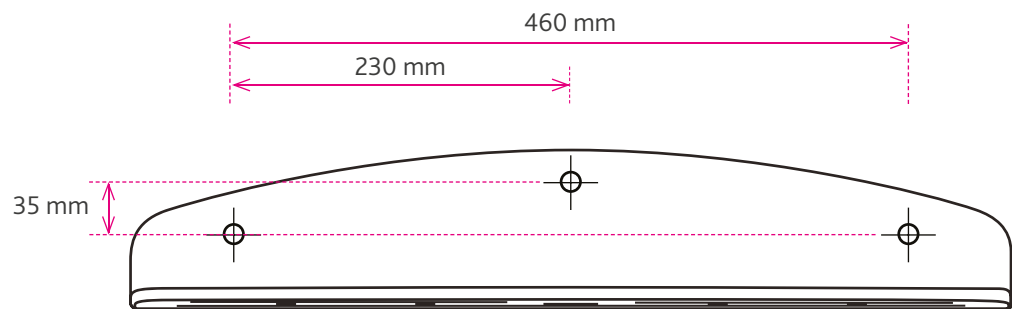
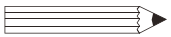
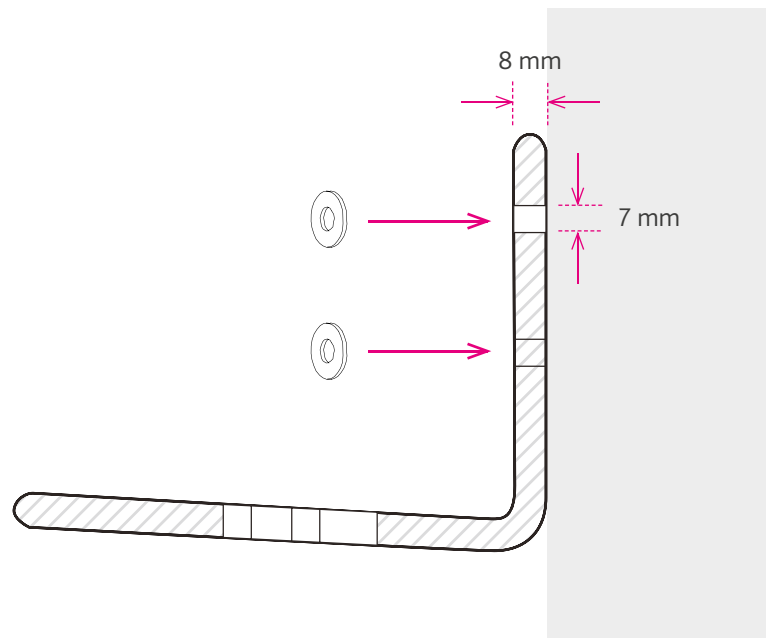
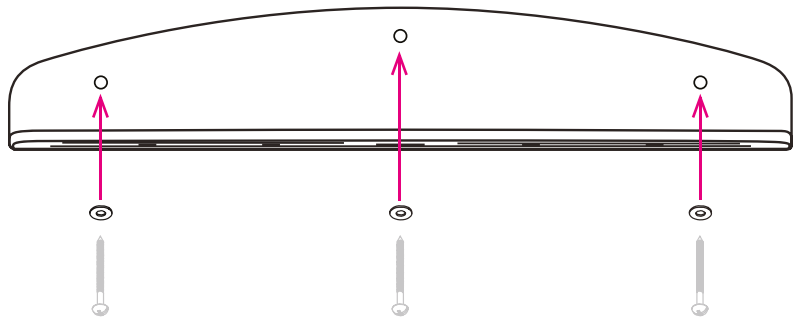
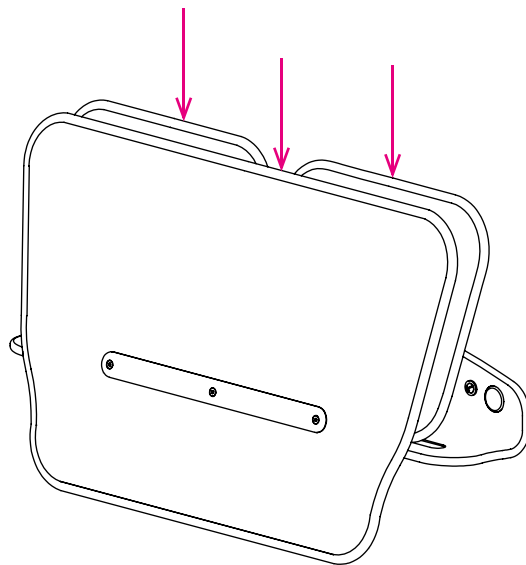
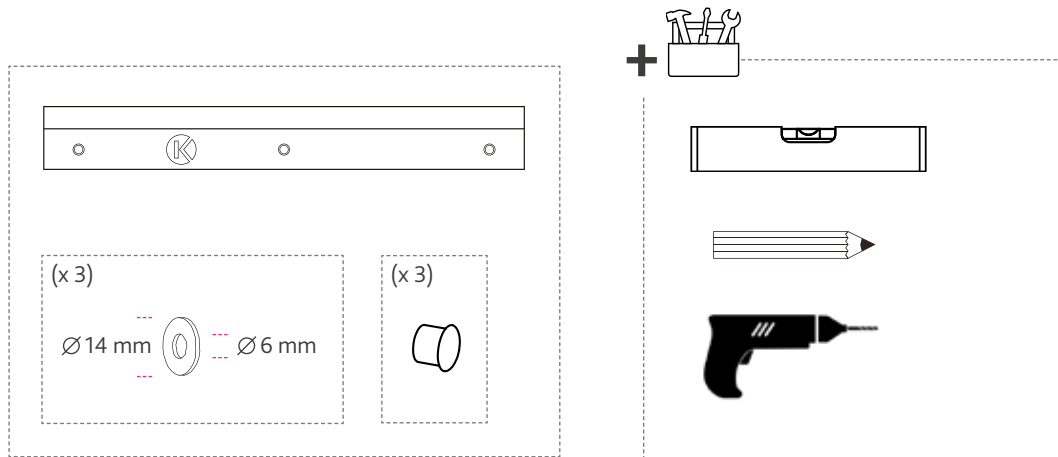
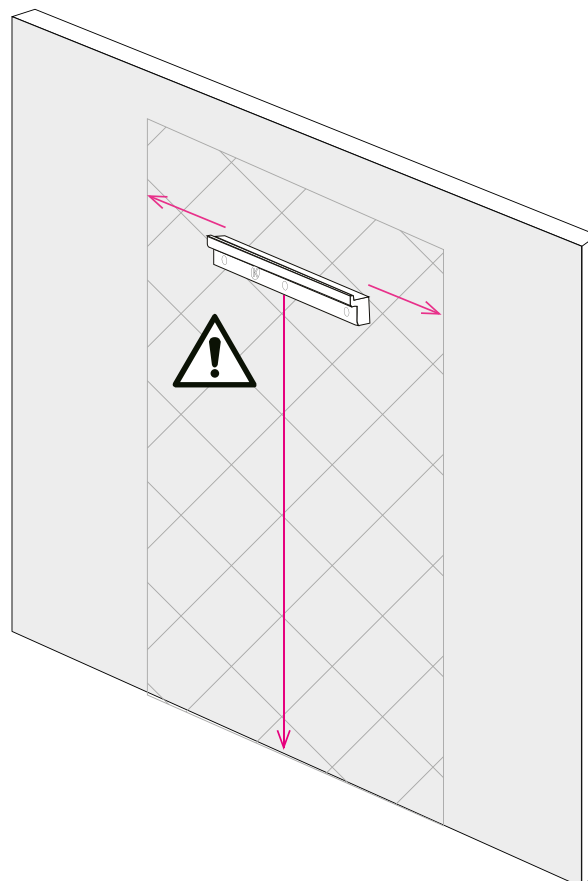
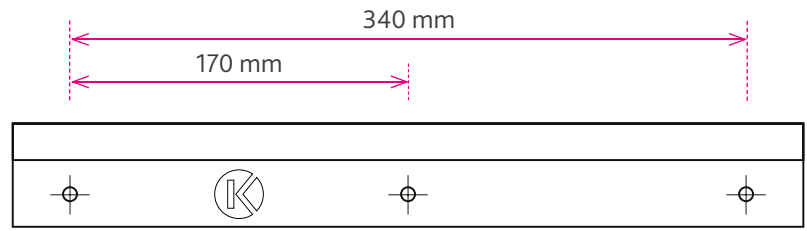
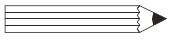
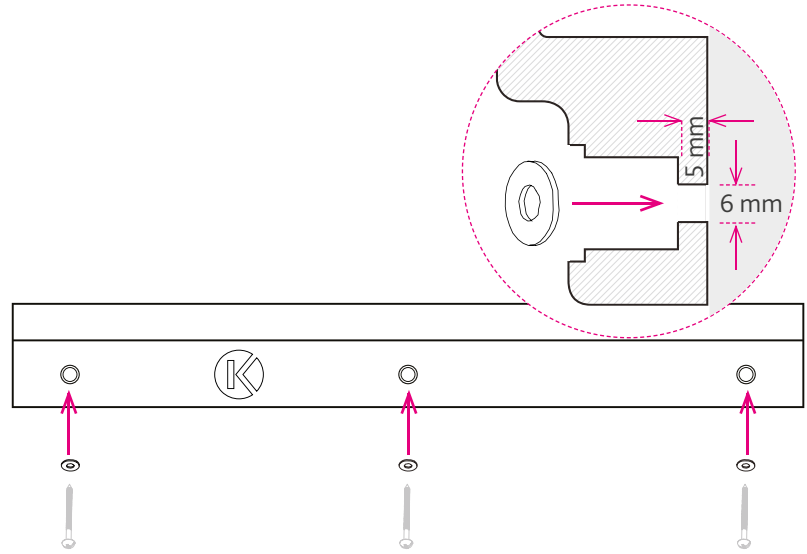
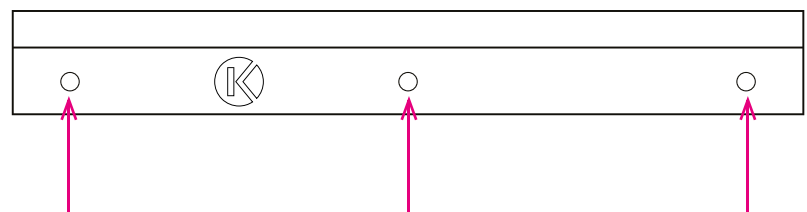
5/5

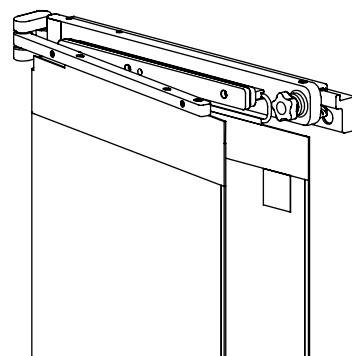max
22 kg

Quality management systems and certification

Kenex is compliant and certified according to the following regulations and standards regarding design and manufacture of medical devices: -

2017/745 Medical device regulation (MDR)
2016/425 Personal protective equipment (PPE)
BS EN ISO 13485:2016 (Quality management systems for medical devices)
BS EN 61331-1:2014 (Determination of x-ray attenuation properties)
BS EN 61331-2:2014 (Translucent protective plates)
BS EN 61331-3:2014 (X-ray protective clothing)
BS EN 60601-1:2021 (Medical electrical equipment – basic safety and essential performance)

Quality management systems audited by SGS UK Limited.

Kenex (Electro-Medical) Limited
Unit 17, RO24 Greenway
Harlow Business Park
Harlow, Essex
CM19 5QB
England

T: +44 (0)1279 417241
E: kenex@kenex.co.uk
www.kenex.co.uk

EC REP

EU authorised representative:

Emergo Europe, Westervoortsedijk 60, 6827 AT Arnhem,
The Netherlands.

Emergo Single Registration Number (SRN): NL-AR-000000116

Kenex Single Registration Number (SRN): GB-MF-000009022

CH REP

Swiss authorised representative:

MedEnvoy Switzerland, Gotthardstrasse 28, 6302 Zug, Switzerland.

Australian Sponsor

Emergo Australia, Level 20, Tower II, Darling Park,
201 Sussex Street, Sydney, NSW 2000, Australia.

US FDA AGENT

Emergo Global Representation LLC, 2500 Bee Cave Road, Building I,
Suite 300, Austin, Texas 78746, USA.